


Instructions

Aromatherapy/Armrest Shelf

Included Parts

Aromatherapy
Armrest Shelf

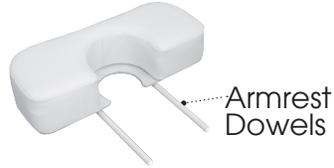

Metal Armrest
Backrest



Aromatherapy
Bowl



Velcro®



Fig. 1



Fig. 2

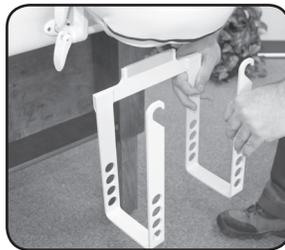


Fig. 3



Fig. 4



Fig. 5



Fig. 6



Fig. 7



Fig. 7A



Fig. 8

Instructions for installing the Armrest Shelf:

- 1) Remove backing from included Velcro® and attach to underside of the table top at the location shown (Fig. 1). Center Velcro® between the two rubber strips and locate the leading edge 2¼ inches from the inside of the frame.
- 2) Insert "Quick Lock™" Face Rest rods into the table-top outlet holes. (Fig. 2)
- 3) Take metal armrest bracket and hook ends of bracket around the facerest rods. Between camlock on facerest and outlet grommets on table. (Fig. 3)
- 4) Tilt metal armrest bracket until the attached Velcro® comes in contact with the Velcro® on the underside of the table. (Fig. 4) This completes the armrest bracket installation.
- 5) Take the aromatherapy armrest shelf and insert into one of the hole sets on the metal armrest bracket. (Fig. 5) Be sure to place armrest dowels in equal holes to maintain level positioning.
- 6) Adjust the armrest shelf height up or down as required for the comfort of the customer. (Fig. 6)

Aromatherapy Use:

- 1) Insert aromatherapy bowl. (Fig. 7) Fill with your choice of essential oil or natural element for client comfort.
- 2) For cleaning; the pad is attached to the wood shelf with Velcro® strips. The pad can be removed by pulling up on the pad. (Fig. 7A) Clean with a mild detergent. Refer to OAKWORKS® upholstery care instructions for details.

Table Extender Use:

- 1) If desired the armrest can be used as a table extender. Insert the armrest dowels directly into the facerest outlet grommets on the end of the table. (Fig. 8)