

INSTRUCTION MANUAL



Celesta Gemini

OAKWORKS®

Celesta Gemini™



*Celesta Gemini
Clodagh Model*



Clodagh Model
WITH FLAT TOP

Manual Part # MMMNST0002
Edition 3, English
August, 2009

All rights are reserved. No part of this document may be photocopied, reproduced or translated to another language without prior written consent of OAKWORKS®, Inc.

OAKWORKS® is a registered trademark of OAKWORKS®, Inc.

Notice

The information contained in this document is subject to change without notice and should not be construed as a commitment by OAKWORKS®, Inc.

OAKWORKS®, Inc. assumes no responsibility for any errors that may appear in this document nor does it make expressed or implied warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

OAKWORKS®, Inc. shall not be liable for incidental or consequential damages in connection with or arising out of the furnishing, performance, or use of this document and the program material which it describes.

TABLE OF CONTENTS

Symbol Identification	pg iii
Warnings	pg iv
Grounding Instructions	pg vi
Section I: Product Description & Photo	
<i>Celesta Gemini</i> [™]	pg 1
<i>Celesta Gemini</i> [™] <i>CLODAGH MODEL</i>	pg 2
Section II: Installation	
Leveling Instructions	pg 3
Section III: Directions for Use	
Adjusting the Height (Foot Control).....	pg 4
Adjusting the Top (Hand Control).....	pg 5
Under-Table Cabinet	pg 6
Side Arm Rests	pg 7
QuickLock [™] Face Rest	pg 9
Accessory Option: Table Extender	pg 11
Moving the table	pg 11
Section IV: Troubleshooting	
Section V: Care & Cleaning	
Section VI: Product Identification	
Section VII: Table Specifications	
Contact Information	back cover



SYMBOL IDENTIFICATION

SYMBOL IDENTIFICATION



This symbol, when used in documentation on this product, represents a caution. Be sure to read and comply with all precautions and warnings.



This symbol, when used in documentation on this product, indicates danger and warns against an electrical shock hazard. Be sure to observe and comply with all warnings and instructions.



This symbol, when used in documentation on this product, indicates a Protective Earth (Ground) Terminal.



This symbol when used in documentation on this product, warns that during transport there should be no stacking of containers.



This symbol, when used in documentation on this product, indicates that the product should be protected from moisture. The humidity specifications for Transport & Storage are listed in the back of this manual.



This symbol, when used in documentation on this product indicates that information is given regarding the recommended temperature limits during transport and storing. The specifications for these limits are listed in the back of this manual.



This symbol, when used in documentation on this product, indicates the date of manufacture of the device.



This symbol, when used in documentation on this product indicates alternating current (AC).



This symbol, when used in documentation on this product, indicates direct current (DC).



This symbol, when used in documentation on this product, indicates that instructional information is included. All instructions should be read carefully and completely before using the equipment.

WARNINGS & IMPORTANT SAFETY INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical furnishing, basic precautions should always be followed, including the following:



Improper use of this device can cause injury. Be sure to read all operating instructions prior to use to prevent injury to operator or client.



Plug table into an outlet that is surge protected to a level of 750J or higher. Failure to do this may void all or part of the warranty.



Electrical Shock Hazard. To reduce the risk of electric shock, always disconnect the table from the electrical outlet before moving the table or doing any maintenance or troubleshooting procedures. Never use this equipment if it has a damaged cord or plug. Use this equipment only for its intended use as described in these instructions. Do not use any attachment or accessories not recommended by the manufacturer.



To reduce the risk of burns, fire, electrical shock or injury:

- The power supply/control module is located at the head end of the table. Refer servicing to qualified personnel and follow manual instructions when doing any authorized troubleshooting. Before doing any work on the unit, disconnect the power cord from the power outlet.
- Before cleaning with any liquid cleaner be sure to un-plug the power cord from the power outlet. Do not operate the table in standing water.
- Before moving the table, disconnect the power cord from the power outlet and secure the electrical cord, hand control and foot controls out of the way for moving.
- *Risk of Electrical Shock.* — Connect this table to a properly grounded outlet only. See Grounding Instructions.
- Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
- Never operate this table if it has a damaged cord or plug., if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the furnishing to a service center for examination and repair.
- Keep the cord away from heated surfaces.
- Do not use outdoors.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.



Risk of Injury — Keep children from moving headrest and foot support.



Never drop or insert any object into any opening.

WARNINGS & IMPORTANT SAFETY INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS (CONT.)



This product is equipped with an interior accessory outlet. The total electrical load of equipment plugged into this set of outlets is not to exceed 840 watts.



Close supervision is necessary when this furnishing is used by, or near, children, invalids or disabled persons. **DO NOT** leave children alone or allow them to operate the controls. Keep controls out of reach of children. **DO NOT** allow children to crawl into the cabinet or under the table.



Be certain that the table is completely lowered without any tilt being present prior to discharging an ambulatory patient. The patient may lose balance and fall. Do not leave clients unattended when table is set at its highest level.



The table weighs as much as 300 lbs. (136 kg.) **DO NOT** lift or move the unit without using proper safety procedures or equipment. **DO NOT** turn the table on its side for any reason. To do so could cause damage to the base and will void the warranty. There are no serviceable items that require this action. Never lift the table by grasping the table top as the table top is not designed to hold the weight of the base and you could damage the table.



When operating the table controls and making any and all adjustments to the table, move hands and feet away from the working mechanism of the table. Use care when opening the doors all the way so that you do not pinch or injure your fingers. Either arm rest can be repositioned to allow a client to get on and off the table easily. When making adjustments to the arm rests make certain that the client's arms are away from the working mechanism.



DO NOT sit on the Arm Rests, the elevated Back Rest or Leg Rest. **DO NOT** place excess weight on the Side Arm Rests. **DO NOT** use them in assisting the client on and off the table.



DO NOT put excessive weight or pressure on the QuickLock™ Face Rest. Excessive pressure can cause premature failure of the QuickLock™ Face Rest. Do Not extend the dowels of the Face Rest further than 3" from the table edge to ensure a stable connection.



DO NOT use the hand control and the foot control simultaneously as this can cause damage to one or both units.



DO NOT store or place anything on the cabinet hood under the backrest as this can cause damage to top and/or hood and will void the warranty.



Maximum Working Load Rating (client and accessories): 625 lbs.

WARNINGS & IMPORTANT SAFETY INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS (CONT.)

WARNING

- *To remove the accessory from the cabinet:* Turn the accessory controls to the 'off' position and then disconnect the unit from the interior power source by removing the plug from the outlet.

When moving the table: Turn the accessory controls to the 'off' position, disconnect the table from the exterior power source by removing the table plug from the outlet.

GROUNDING INSTRUCTIONS




GROUNDING INSTRUCTIONS:

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electrical current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

This product is for use on a nominal 120-volt circuit (see below, USA plug example) and has a grounding plug that looks like the plug illustrated in Figure A. Make sure that the product is connected to an outlet having the same configuration as the plug.

No adapter should be used with this product.

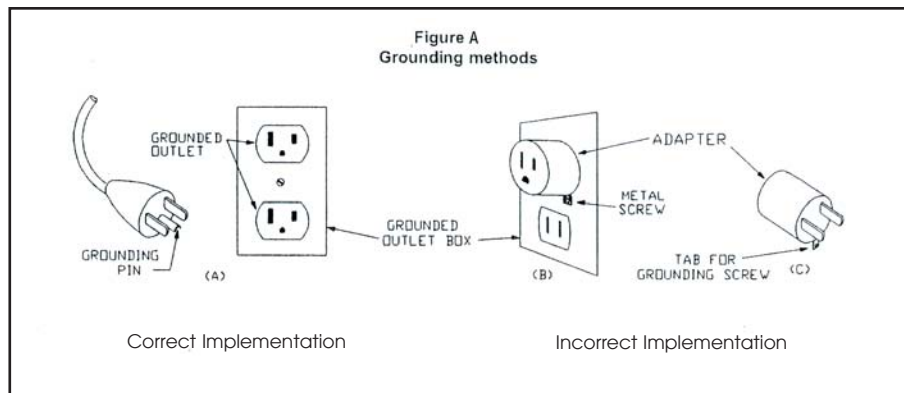

DANGER-Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or service man if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product - if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

PRODUCT DESCRIPTION

Celesta Gemini™

PRODUCT DESCRIPTION

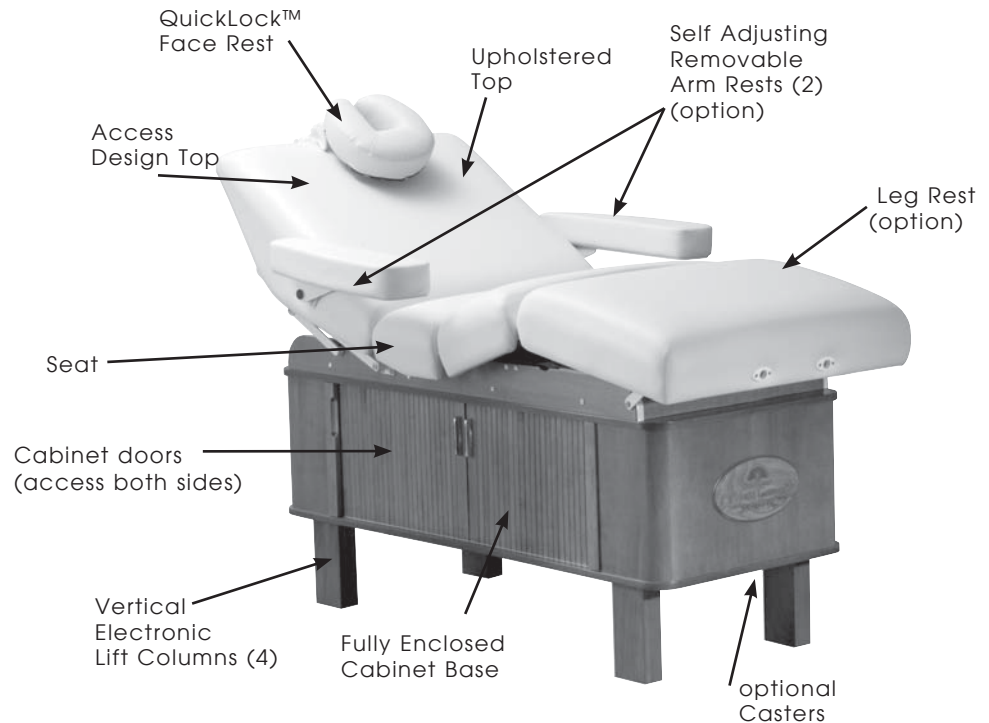


Table Specs* :

- Height* : 25" - 37" (64 cm - 94 cm)
- Width: 29" (74cm), 31" (79cm), or 33" (84 cm)
- Length: Flex top - 72" (183 cm)
Flat top - 73" (185 cm)
- Lift Capacity: 625 lbs (284 kg)
Maximum Working Load
- Weight: 300 lbs. (136 kg)
- Top: Electronically Controlled
Access Design Top & FlatTop
- Electronics: 1 Hand Control (flex top only)
& 1 Foot Control
- Padding: Aero·Cel™ System; 4" or 6"
- Upholstery: Ultratouch II or TerraTouch™
- Cabinetry: Wood/Baltic Birch

For electrical specs, see Specifications Chart.

* measurements are determined from floor to top of seat pad using standard padding; these measurements can be affected by natural material variations

** published weight may differ due to options chosen, natural material variations

PRODUCT DESCRIPTION

Celesta Gemini™ CLODAGH MODEL

PRODUCT DESCRIPTION

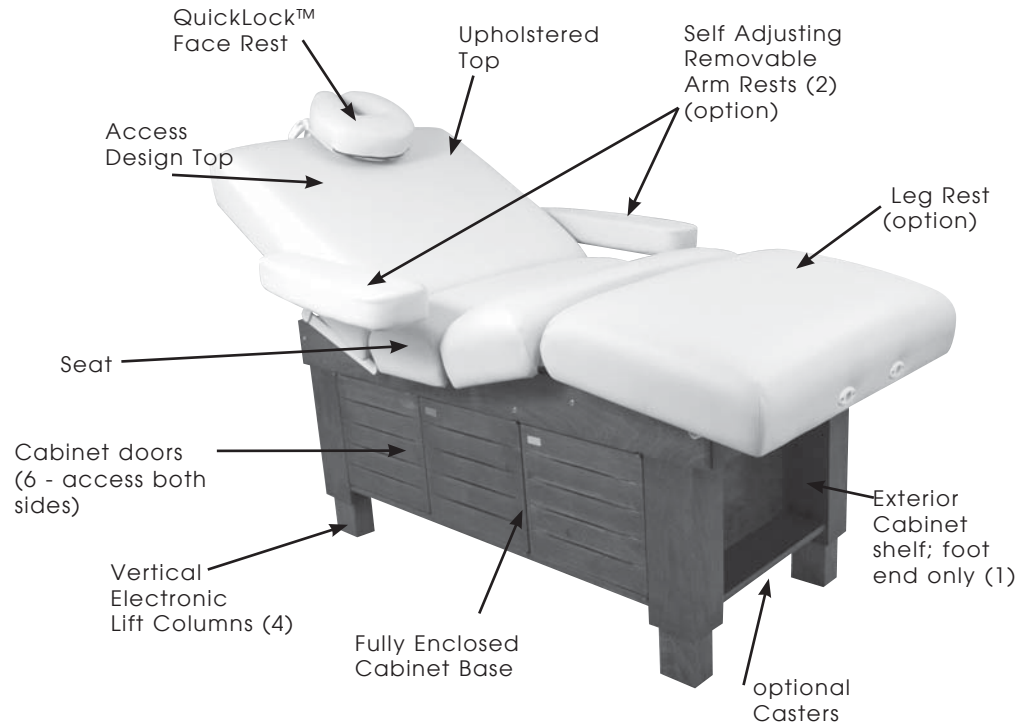


Table Specs* :

- Height *: 25" - 37" (64 cm - 94 cm)
- Width: 29" (74cm), 31" (79cm), or 33" (84 cm)
- Length: Flex top - 72" (183 cm)
Flat top - 73" (185 cm)
- Lift Capacity: 625 lbs (284 kg)
Maximum Working Load
- Weight: 300 lbs. (136 kg)
- Top: Electronically Controlled
Access Design Top & FlatTop
- Electronics: 1 Hand Control (flex top only)
& 1 Foot Control
- Padding: Aero·Cel™ System; 4" or 6"
- Upholstery: Ultratouch II or TerraTouch™
- Cabinetry: Wood/Baltic Birch

For electrical specs, see Specifications Chart.

* measurements are determined from floor to top of seat pad using standard padding; these measurements can be affected by natural material variations

** published weight may differ due to options chosen, natural material variations

INSTALLATION

• Leveling Feet



When operating the table controls be sure to observe all cautions and warnings given in the manual to prevent injury to both operator and client.

The table comes completely assembled and ready to use. Plug the cord into a functioning outlet that is rated for the table.

Arrange the power cord and control cords so that they will not create a tripping hazard. Place the controls in a safe location where they are convenient and accessible. Keep the controls out of the way of all table mechanisms.

LEVELING INSTRUCTIONS:

If Table Will Not Sit Flat on the Floor:



The table is equipped with Leveler Feet that adjust to level the table on uneven floors. To adjust the Leveler Feet to suit common floor differences you will need to use a 1/2" open-end wrench or an adjustable wrench. Adjust each leg individually in turn by adjusting the height of each leveler foot until the unit rests level and steady. This adjustment will not effect the function of the table.

▶▶ IMPORTANT



CAUTION

The table weighs as much as 300 lbs. (136 kg.) *DO NOT* lift or move the unit without using proper safety procedures or equipment. *DO NOT* turn the table on its side for any reason. To do so could cause damage to the base and *will* void the warranty. There are no serviceable items that require this action. Never lift the table by grasping the table top as the table top is not designed to hold the weight of the base and you could damage the table.

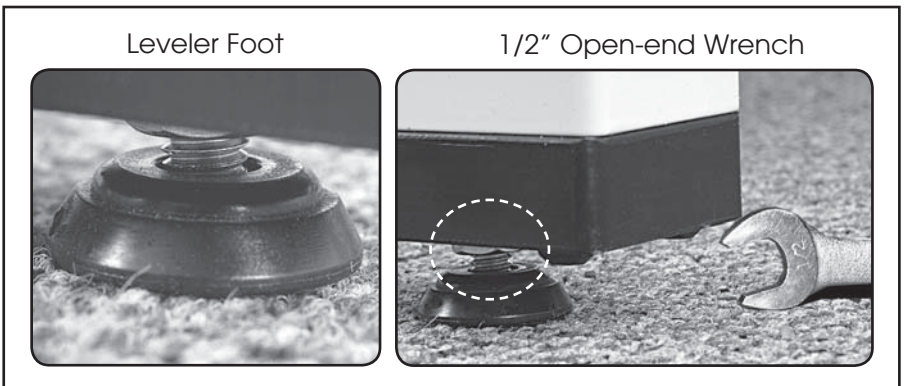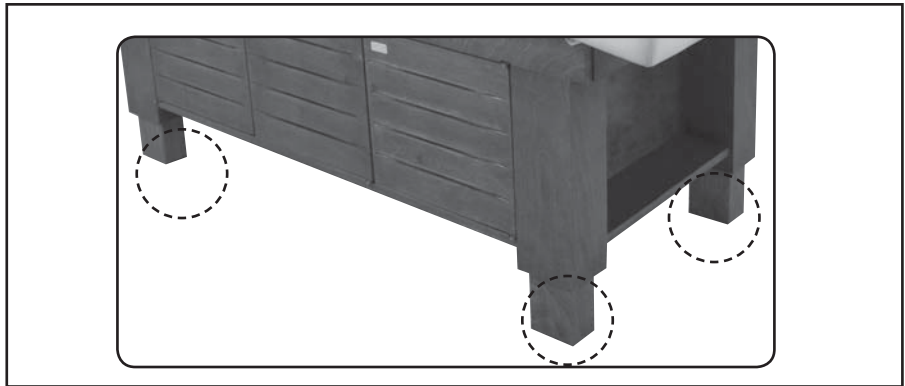

DIRECTIONS FOR USE


ADJUSTING HEIGHT OF THE TABLE:

- Foot Control

CAUTIONS

When operating the controls and making any and all adjustments to the table, move hands, feet and controls away from any working mechanisms of the table.


Children must be supervised when around this equipment by a responsible adult. Do not leave children alone or allow them to operate the controls. Do not allow children to crawl into the cabinet or under the table.

 When operating the controls be sure to observe all cautions and warnings given in the manual to prevent injury to both operator and client.

The table comes complete with a multifunctional hand control that operates the backrest and leg rest positioning functions as well as a foot control that operates the height with electronic ease. These controls should not be used simultaneously but individually to ensure safety and complete ease of use.

Foot Controls:

Operate the foot control as shown below to raise or lower the height of the *Celesta Gemini*™. The foot control allows the therapist to adjust the height function separately from the other functions found on the hand-control whenever needed. When operating the table's controls, be sure to observe all cautions and warnings as given in the manual.

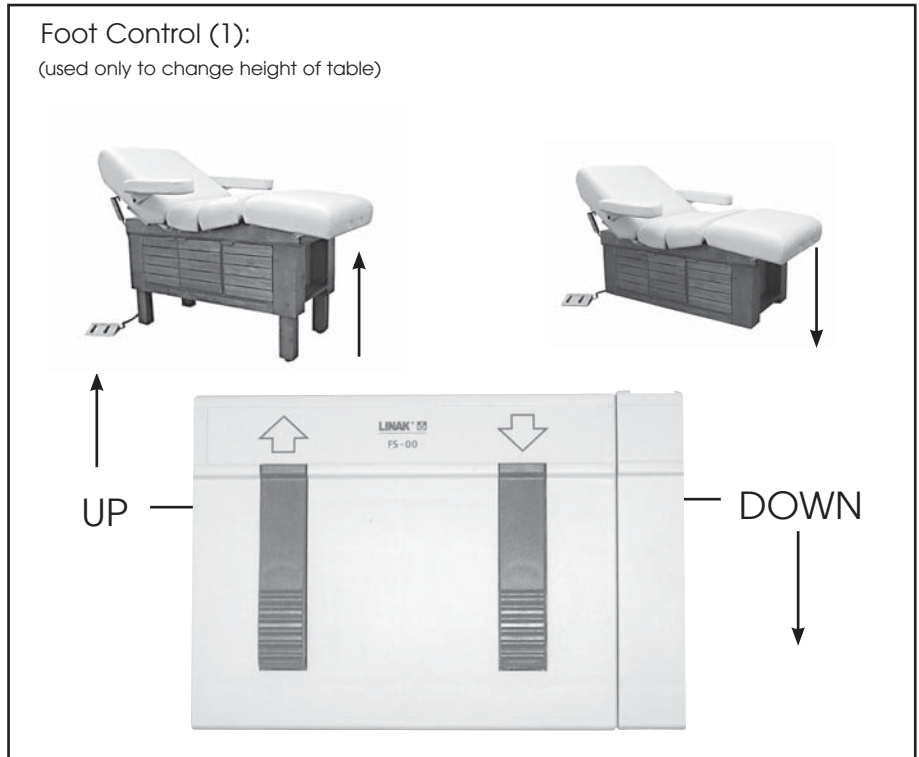

▶▶ IMPORTANT

CAUTION

DO NOT use the hand control and the foot control simultaneously as this can cause damage to one or both units.

DIRECTIONS FOR USE

HAND CONTROL OPERATIONS (FLEX TOP MODELS ONLY)

• Hand Control




When operating the controls, be sure to observe all cautions and warnings given in the manual to prevent injury to both operator and client.



CAUTIONS

Do not sit on the elevated Back Rest or Leg Rest.

Children must be supervised when around this equipment by a responsible adult. Do not leave children alone or allow them to operate the controls. Do not allow children to crawl into the cabinet or under the table.

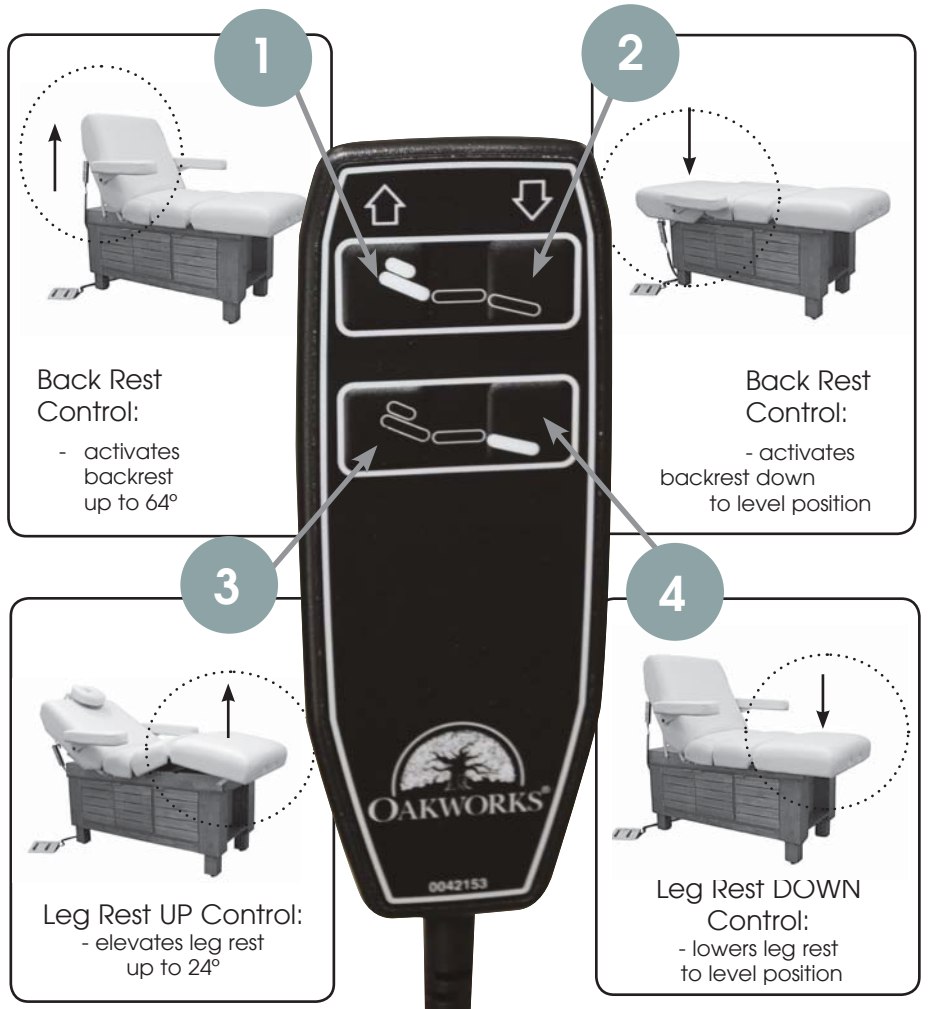

▶▶ IMPORTANT



CAUTION

DO NOT use the hand control and the foot control simultaneously as this can cause damage to one or both units.

Press and hold the appropriate button until the position desired is achieved and then release. When accessing controls 3 & 4, the leg position will move in tandem with the thigh section and maintain the angle of original setting. The arm rests will adjust automatically to provide the most in comfort and ergonomics while staying parallel to the floor, no matter what the position.

DIRECTIONS FOR USE

▶▶ IMPORTANT

CAUTION

DO NOT store or place anything on the cabinet hood under the backrest as this can cause damage to the top and/or hood.

IMPORTANT SAFETY NOTE

The cabinet hood area under the backrest top is not designed for storage or as an area to hold accessories while with a client. DO NOT use this area for holding *anything*. Damage can occur to the table, especially when the pre-set position commands that are accessed by the hand control are activated. Placing anything on this area is prohibiting movement of the backrest and can cause damage to the table or the object. Using this area as storage will void the warranty.



CAUTION

Use care when opening the doors all the way so that you do not pinch or injure your fingers.

Children must always be supervised when around this equipment by a responsible adult. Do not leave children alone or allow them to operate the controls. Do not allow children to crawl into the cabinet or under the table.

DANGER



This product is equipped with an interior 120-volt 60Hz accessory outlet. Disconnect all power cords from the interior power outlet (found within the cabinet) after use. The total electrical load of equipment plugged into this set of outlets is not to exceed 7 amps (840 watts).

▶▶ *Note: Appearance of outlets will vary depending on country of use.*

UNDER TABLE CABINET

To Access the Storage in the Under Table Cabinet:

The table has a fully integrated beautiful wood cabinet and base. The large under-table cabinet provides storage for stacks of towels and other equipment and an in-cabinet plug to accommodate hot towel cabinets and other accessories.

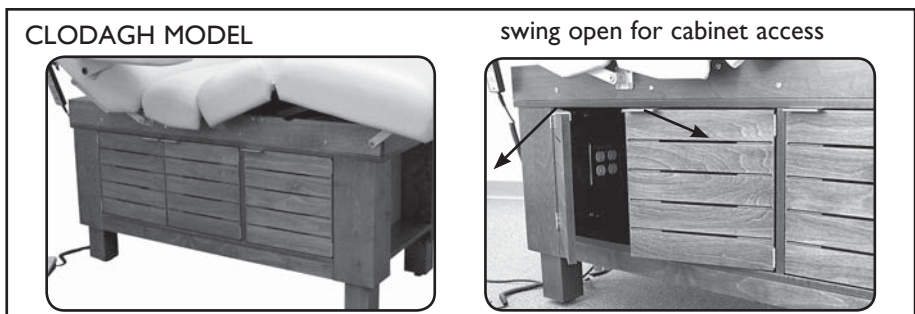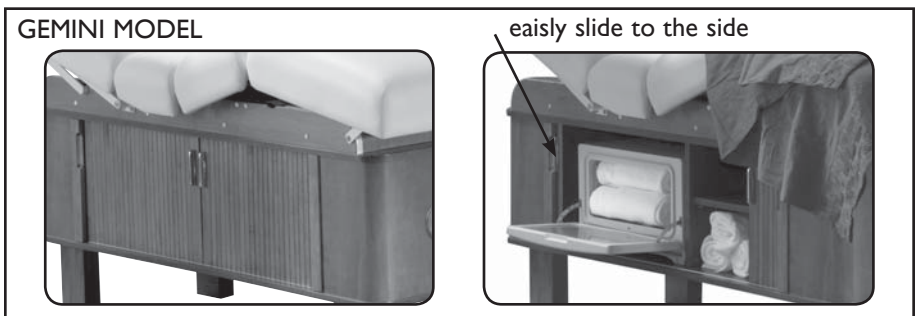

DIRECTIONS FOR USE

• Side Arm Rests

▶▶ IMPORTANT



CAUTION

Do not sit on the Arm Rests.

Do not place excess weight on the Side Arm Rests. Do not use them in assisting the client on and off the table.

Either Arm Rest can be repositioned to allow a client to get on and off the table easily.

When making adjustments the arm rests make certain that the client's arms are away from the working mechanism.



Be sure to read all cautions, warnings and instructions given in the manual to prevent injury to both operator and client.

SIDE ARM RESTS

Though the arm rests adjust automatically to remain parallel to the floor in any position, there will still be times when you will need to move an arm rest out of the way or remove them altogether. We recommend that you raise the arm rest out of the way to allow a client to get on or off the table. They are designed to be strong and will take a reasonable amount of weight but they are not designed to be used to sustain a person's weight when getting on or off the table.

To Adjust the Arm Rest:

Before asking a client to mount or dismount from the table we recommend that you move the arm rest out of the way. The arm rests are easy to move and manually adjust. First position the tabletop with the backrest in the upright position and all other section flat. Take hold of the arm rest firmly and raise the arm rest up and out of the way with a steady swing until the arm rest is in the highest upright position (see images below). Hold it in place until your client has mounted (or dismounted) and then actively move it back into the resting position where it will proceed to move automatically with the table top.

Adjusting the Side Arm Rest



The arm rest is now set and will adjust automatically as you position the client for whatever procedure is needed.

There will be times when removal of the arm rests will be needed. This feature is helpful when doing massage or helping someone get on or off the table. This can be done easily and storage is conveniently located in the under-table cabinet. They are then at hand when you are ready to re-install.

To Remove the Arm Rest:

The arm rest frame has a notched connection to the interior of the table top which allows it to move automatically when the table top changes position. (cont. on next page)

DIRECTIONS FOR USE

• Side Arm Rests (cont.)

IMPORTANT



CAUTION

Do not sit on the Arm Rests.

Do not place excess weight on the Side Arm Rests. Do not use them in assisting the client on and off the table.

Either Arm Rest can be repositioned to allow a client to get on and off the table easily.

When making adjustments the arm rests make certain that The client's arms are away from the working mechanism.



CAUTION

Be sure the arm rest is fully inserted before lowering.

SIDE ARM RESTS: (CONT.)

There will be times when removal of the arm rests will be needed. This feature is helpful when doing massage or helping someone get on or off the table.

To Remove the Arm Rest:

To remove one or both arm rests just take hold of the base of the arm rest at the metal frame and somewhere on the cushion. Holding securely, rotate the arm rest while at the same time pulling slightly away from the table. You will feel the notched connection slide out as you rotate the arm rest. When you feel this you can then slide the entire arm rest out of the connection hole very easily. Store it safely until ready for installation.



Re-attach the armrest by sliding the arm rest frame into the hole on the backrest. Push it in securely and allow the arm rest to rest in the parallel position. This resets the arm rest in the proper position for the automatic adjustment feature to be activated.

Removing the Side Arm Rests



MANICURE ARM RESTS:

The manicure arm rest installation follows the same standard installation and removal instructions as above.

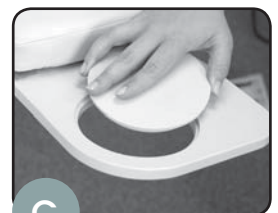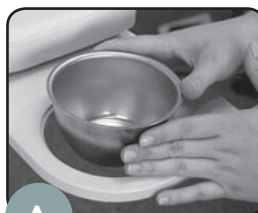

- A. hand soak bowl installation
- B. hand soak bowl removal
- C. Manicure surface installation



DIRECTIONS FOR USE

- Accessory Option:
QuickLock™
Face Rest

QuickLock™ FACE REST:

Double articulating action provides nearly limitless positioning options with the QuickLock™ Face Rest. The adjustable pad takes pressure off sensitive facial areas while providing superb support. The QuickLock™ Face Rest can be used in both prone and supine positioning and folds out of the way when not in use. Adjustments are made easily by loosening the cam locks, adjusting for height and angle and then locking the cam locks into that position. The table has webbing loops at the head of the table for a variety of uses and outlets at the other end that work as well for additional procedures.

▶▶ IMPORTANT




CAUTION

Do *Not* extend the dowels of the Face Rest further than 3" from the table edge to ensure a stable connection.



Be sure to read all cautions, warnings and instructions given in the manual to prevent injury to both operator and client.

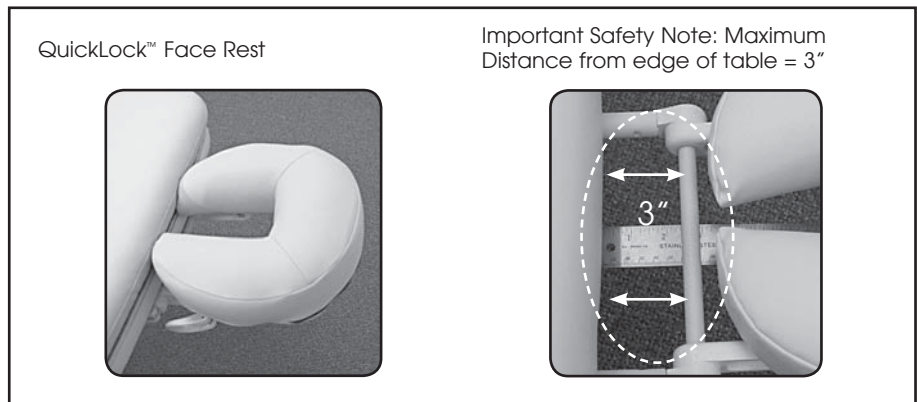

Attaching the QuickLock™ Face Rest / Head End Webbing:

When attaching the QuickLock™ Face Rest, just slide the frame dowels into the webbing loops, being sure to keep the 1st cam lock within 3" of the table to ensure a stable connection. The recommended length of dowel bracing against the backrest should be at least 2 inches.


To remove the QuickLock™ Face Rest, just pull the frame from the webbing loops until it comes free. Set it safely aside or store it inside the under-table cabinet until needed.

▶▶ IMPORTANT



CAUTION

Do not put excessive weight or pressure on the face rest. Excessive pressure can cause premature failure of the QuickLock™ Face Rest.

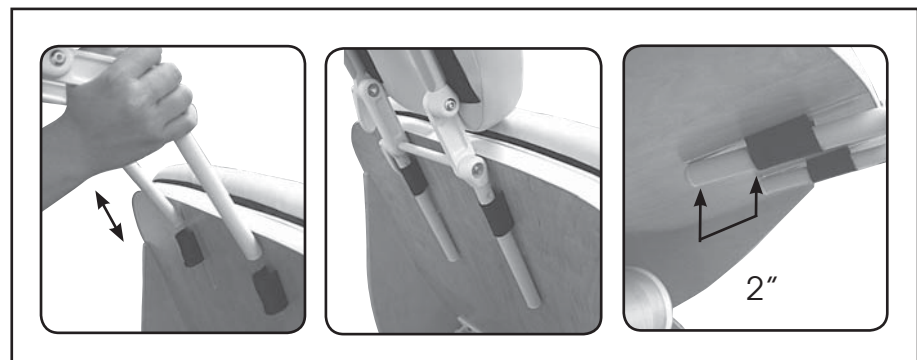

* U.S. Patent #5,177,823

DIRECTIONS FOR USE

- Accessory Option:
QuickLock™
Face Rest (cont.)




CAUTION

Do not put excessive weight or pressure on the face rest. Excessive pressure can cause premature failure of the QuickLock™ Face Rest

Attaching the QuickLock™ Face Rest on on Flat Top models & Flex Top Foot End Outlets:

When attaching the QuickLock™ Face Rest to the foot end by using the provided outlets, just slide the frame dowels into the outlets being sure to keep the 1st cam lock within 3” of the table to ensure a stable connection.

To remove the QuickLock™ Face Rest, just pull the frame from the outlet until it comes free. Set it safely aside or store it inside the under-table cabinet until needed.

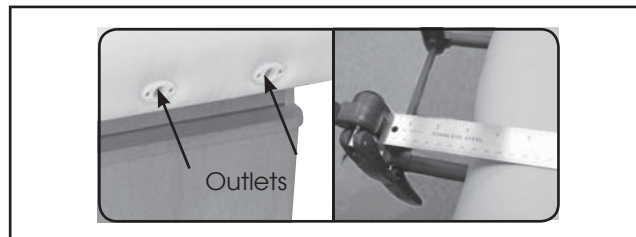


Tips for Great Positioning using the QuickLock™ Face Rest:

- To use the face rest with your clients supine, rotate the face rest pad so that the thick part of the pad supports your client’s cervical region. Adjusting the height and angle of the face rest gives your clients perfectly comfortable support.



- The face pad is flexible so that you can bring the open ends together or spread them apart to achieve the most comfortable support for any size face. To avoid your client’s chin from hitting the spacebar, make sure the pad is narrow enough to support their whole face.

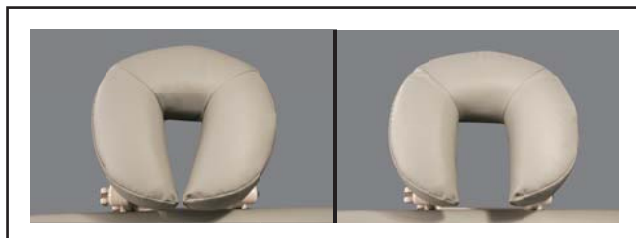

- For appropriate cervical flexion, use the cam locks to adjust the downward angle of the face rest, making sure that the surface of the face rest pad at the chin is not lower than the surface of the table.

* U.S. Patent #5,177,823

DIRECTIONS FOR USE

• Accessory Option: Table Extender




Be sure to read all cautions, warnings and instructions given in the manual to prevent injury to both operator and client.



CAUTION

Do not put excessive weight or pressure on the extender. Excessive pressure can cause premature failure of the Table Extender.

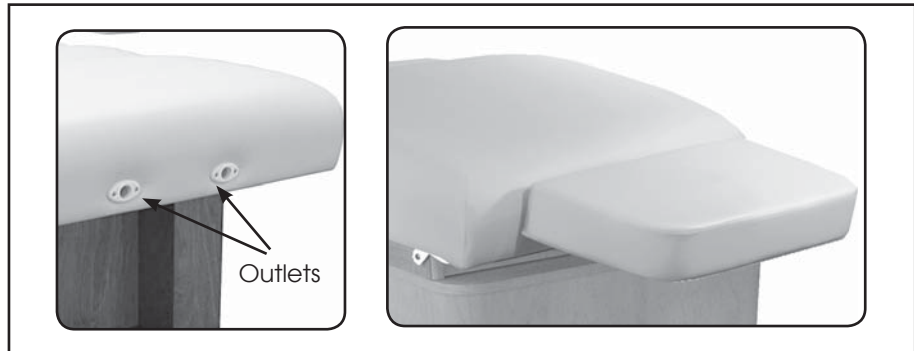

The optional stationary Table Extender provides ultimate comfort for your tallest clients, quickly and easily. The Table Extender comes with two connecting arms that slide into the outlets at the foot end of the tabletop. Slide these fully into the outlets so the Table Extender is flush to the end of the table top. This Stationary Extender provides 11" of additional comfort and support when needed.

Moving the table




CAUTION

The table weighs as much as 300 lbs. (136 kg.) *DO NOT* lift or move the unit without using proper safety procedures or equipment. *DO NOT* turn the table its side for any reason. To do so could cause damage to the base and *will* void the warranty. There are no serviceable items that require this action. Never lift the table by grasping the table top as the table top is not designed to hold the weight of the base and you could damage the table.



CAUTION

Before moving the table, disconnect the power cord from the power outlet and secure the electrical cord, hand control and foot controls for moving.

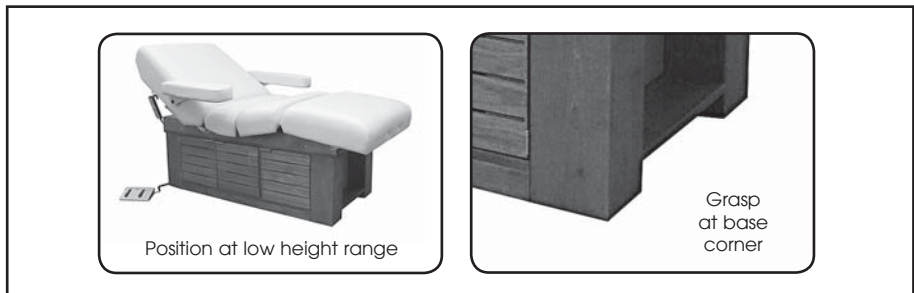

The table weighs approximately 300 lbs. (136 kg.) You will need 4 people working together if it is necessary to move the table. Never lift the table by grasping the table top as the table top is not designed to hold the weight of the base and you could damage the table and the electronics that run the tabletop.

If you need to move the table, place one person at each corner of the table. We recommend that you have the table top in the up position (with Arm Rests removed) to have the table top ends out of the way as much as possible. Have each person firmly grasp the cabinet base at the bottom by the table legs and lift the table from the base. As when moving any piece of heavy equipment, bend at the knee and lift with your legs beneath you and not with your back. Be aware and use proper, safe procedures for moving a table of this size to prevent injury to yourself or the table. We strongly recommend that a flatbed dolly be used that can hold the entire table if moving the table involves any distance.

TROUBLESHOOTING

▶▶ IMPORTANT



CAUTION

The table weighs as much as 300 lbs. (136 kg) *DO NOT* lift or move the unit without using proper safety procedures or equipment. *DO NOT* turn the table on its side for any reason. To do so could cause damage to the base and *will* void the warranty. There are no serviceable items that require this action. Never lift the table by grasping the table top as the table top is not designed to hold the weight of the base and you could damage the table.



Be sure to read all cautions, warnings and instructions given in the manual to prevent injury to both operator and client.

IF THE TABLE NOT CHANGE HEIGHT OR POSITION:

- Check the outlet to be sure that it has power and that the power cable is plugged in. Try each Foot Control and Hand Control to see if any work. If any work, then possibly there is a loose connection or a faulty controller.
- Unplug the power cable from the wall outlet.
- Make sure that all male and female connectors are firmly pushed into each other. Even a slight misconnection can cause a possible malfunction.
- Plug the unit back into the wall and check the controls again. If none work, then contact customer service.

Questions to ask before calling Customer Service:

- Is the unit plugged in?
- Does the outlet have power?
- Check all connections of the control box, is everything seated securely?
- Does the height of the table work when you press on the foot pedal?
- Does the top work when you use the hand control?

CARE & CLEANING

In order to maintain the quality and ensure the life of the table, here are some preventative steps that can be taken to protect the upholstery on the chair and accessories.

• Upholstery

- Protect your table and accessories with sheets and fleeces. We offer both fitted sheets and thick fleeces that protect your upholstery fabric against oil stains while offering comfortable 100% cotton flannel softness to your clients. You can also preserve your face rest and bolsters with 100% cotton flannel covers and fleeces.
- Clean your upholstery fabric every day. We recommend cleaning your upholstery fabric with a mild detergent to protect it against oil stains and dirt which will cause cracking and permanent stains over time. Household products you can use include a 4:1 diluted solution of 409® cleaner, Fantastik® or any non-abrasive with a non-alcohol content. Products with a high alcohol content, when used over time, will dry and eventually crack your upholstery fabric.

Note: Damage caused by them will not be covered under the warranty.

- Avoid extremes in heat. Very cold temperatures can make your fabric prone to cracking. Very high temperatures can cause the fabric to stretch permanently.
- Avoid exposing your products to direct sunlight for extended periods of time. This can lead to fading and cracking of the fabric.
- Use an anti-slip pad to prevent oil from reaching the upholstery and to keep your sheets from slipping.

GENERAL CARE OF EQUIPMENT COMPONENTS




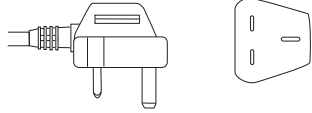
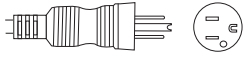
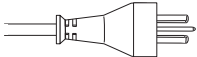
• Base & Controls

- Use only a mild solution of 4:1 diluted non-alcohol cleaner such as 409®, Fantastik® or some other non-abrasive cleaner to wipe down the metal surfaces of the chair. Apply with a damp (not wet) lint-free cloth. Be sure excess liquid does not drip onto, or into, any of the electrical controls or mechanisms. Be sure that the unit is unplugged before cleaning these areas. Wipe these areas carefully with a lint free cloth.
- The electronic powering mechanism of your table requires no general maintenance or lubrication.
- The wooden portions of the equipment can be polished with any polish approved for wooden surfaces. Spray, or pour, the polish onto a soft cloth and be careful not to get it onto other surfaces.

PRODUCT IDENTIFICATION

• Plugs

▶▶ *These part numbers are for identification only; they are not field serviceable items.*

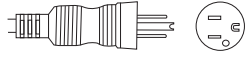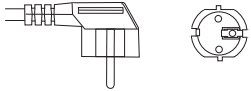OAKWORKS® Part #:	Voltage (ACE)	Power Cord Plug
7791	120v 60 Hz	3 prong grounded hospital grade; North America 
7791-03	220v 50 Hz	Continental Plug; Europe/Italy 
7791-04	220v 50 Hz	Swiss plug; Switzerland 
7791-02	220v 50 Hz	British plug; United Kingdom 
7791-05	100v 50/60 Hz	3 prong polarized plug; Japan & Korea 
7791-06	220v 50 Hz	Australian Plug; Australia 

PRODUCT SPECS

ELECTRICAL SPECIFICATIONS

Designed For:	North America	Europe	Japan/Korea
Electric Service	120VAC/15A	220VAC/10A	100VAC/20A
Input Frequency	60 Hz	50 Hz	50/60 Hz
Voltage Output	24 VDC	24 VDC	24 VDC
Current Consumption w/o internal outlets	4.7 amps	2.6 amps	5.6 amps
Electric Shock Protection	Class I Equipment	Class I Equipment	Class I Equipment
Mode of Operation	Continuous Operation is limited to a 10% Duty Cycle (2 min - 18 min rest)	Continuous Operation is limited to a 10% Duty Cycle (2 min - 18 min rest)	Continuous Operation is limited to a 10% Duty Cycle (2 min - 18 min rest)

DIMENSIONS



Length	Flex 73" Flat 72"	185 cm 183 cm
Width	29", 31", 33"	74 cm, 79 cm, 84 cm
Height Range	25" - 37"	64 - 94 cm
Weight	300 lbs.	136 kg
Shipping Weight	~ 375 lbs.	~ 170 kg
Shipping Crate Dimensions	81" L x 36" H x 35" W	205.7cm x 91.4cm x 89cm

Cabinet Dimensions:

Total Outside Dimensions:	14.8" H x 24.2" W x 56.5" L	37.8cm L x 61.6cm H x 143.5cm W
Storage Area 1/ Head End:	13.5" H x 24.2" W x 21.7" L	34.3cm L x 61.6cm H x 55.2cm W
Storage Area 2/ Foot End:	13.5" H x 24.2" W x 17" L	34.3cm L x 61.6cm H x 43.2cm W
Opening Dimensions: Storage Area 1/ Head End:	13.5" H x 20.1" L	34.3cm H x 51.1cm L
Opening Dimensions: Storage Area 2/ Foot End:	13.5" H x 15.3" L	34.3cm H x 39.1cm L

ACCESSORY OPTIONS

Part No.	Accessory Description
1732 & 1733	Side Armrests (2, includes frame & cushions)
46467	Manicure Arm Rests
0218	QuickLock™ Face Rest (2 parts)
2410	- Platform (1) - Crescent Face Rest Pad(1)
3680-1	Hot Towel Cabi; 120V, US
3680-2	Hot Towel Cabi; 220V, EU

Storage & Transport	 Temperature: 32° - 100° F 0°-38° C	Humidity: 60% relative humidity	 Pressure: no limitations known
	During transport, DO NOT stack containers.		

* measurements are determined from floor to top of seat pad using standard padding; these measurements can be affected by natural material variations in padding.

Celesta Gemini™

INSTRUCTION MANUAL

CONTACT INFORMATION:

OAKWORKS® Inc.

923 East Wellspring Road
New Freedom, PA 17349

Toll Free (USA only): 800-916-4613
Phone: 717-235-6807

FAX: 717-235-6798

www.oakworks.com



European Compliance for CE Information:
Emergo Europe
Molenstraat 15
The Hague, 2513 BH Netherlands
info@emergogroup.com



Manual Part # MMMNST0002
Revision date: 08.06.09
Edition 3, English
Printed in U.S.A.

