

INSTRUCTION MANUAL

OAKWORKS® Lift Tables



Celesta® Lift
ELECTRIC LIFT & TILT
MANUAL TOP



Celesta® Deluxe
ELECTRIC LIFT & TILT
ELECTRIC TOP



Celesta® PerformaLift
ELECTRIC LIFT & TILT
MANUAL TOP

AVAILABLE WITH:

- RECTANGULAR FLAT TOP
- ROUND CORNER FLAT TOP
- BACKREST TOP
- FLEX TOP (SHOWN HERE)



Manual Part Number
MMMNST0003
Revision date: 10.26.09
2nd Edition, October 2009
1st Edition, September 2003
English, Printed in USA

All rights are reserved. No part
of this document may be
photocopied, reproduced or
translated to another language
without prior written consent
of OAKWORKS®, Inc.

OAKWORKS® is a registered
trademark of OAKWORKS®, Inc.

Notice

The information contained in this
document is subject to change
without notice and should not
be construed as a commitment by
OAKWORKS®, Inc.

OAKWORKS®, Inc. assumes no
responsibility for any errors that
may appear in this document nor
does it make expressed or implied
warranty of any kind with regard
to this material, including, but not
limited to, the implied warranties
of merchantability and fitness for
a particular purpose.

OAKWORKS®, Inc. shall not be
liable for incidental or consequen-
tial damages in connection with or
arising out of the furnishing, per-
formance, or use of this document
and the program material which it
describes.

TABLE OF CONTENTS

Symbol Identification	pg 1
Important Safety Instructions	pg 2
Section I: Product Description & Photo	
<i>Celesta® Deluxe</i>	pg 3
<i>Celesta® Lift</i>	pg 4
<i>Celesta® PerformaLift</i>	pg 5
Section II: Installation	
Leveling Instructions	pg 6
Section III: Directions for Use	
Foot Control Operations	
<i>Celesta® Deluxe, Celesta® Lift & Celesta® PerformaLift</i>	pg 8
Hand Control Operations	
<i>Celesta® Lift & Celesta® PerformaLift</i>	pg 9
Hand Control Operations	
<i>Celesta® Deluxe</i>	pg 10 & 11
Manual Top Operations	
<i>Celesta® Lift & Celesta® PerformaLift</i>	pg 12
Grounding	pg 13
Section IV: Accessories	
Side Arm Rests	pg 14 & 15
Lowered Arm Rest Shelf	pg 16
QuickLock™ Face Rest	pg 16
Section V: Troubleshooting	
Thermal Limiting Switch	pg 17
If the table will not change position	pg 17
Questions to ask before calling Customer Service.....	pg 17
Section VI: Care & Cleaning	pg 18
Section VII Product Identification	pg 19
Section VIII: Specifications	pg 20
Contact Information	back cover

SYMBOL IDENTIFICATION



This symbol, when used in this manual and on product labels, represents a caution warning. Be sure to read and comply with all precautions and warnings.



This symbol, when used in this manual and on product labels, warns against an electrical shock hazard. Be sure to observe and comply with all warnings.



This symbol, when used in this manual or on product labels, indicates a Protective Earth (Ground) Terminal.



This symbol when used in this manual or on product labels, warns that during transport there should be no stacking of containers.



This symbol, when used in this manual or on product labels, indicates that the product should be protected from moisture. The humidity specifications for Transport & Storage are listed on page 31.



This symbol, when used in this manual or on product labels, indicates that information is given regarding the recommended temperature limits during transport and storing.



This symbol, when used in this manual or on product labels, indicates the date of manufacture of the device.



This symbol, when used in this manual or on product labels, indicates alternating current (AC).



This symbol, when used in this manual or on product labels, indicates direct current (DC).

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical furnishing, basic precautions should always be followed, including the following:



DANGER - To reduce the risk of electrical shock always unplug this furnishing from the electrical outlet before cleaning.



WARNING - To reduce the risk of burns, fire, electric shock or injury to persons:



1. Unplug from outlet before putting on or taking off parts.



2. Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.



3. Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.



4. Never operate this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Contact Oakworks Customer Service before use.



5. Keep the cord away from heated surfaces.



6. Never drop or insert any object into any opening.



7. Do not use outdoors



8. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.



9. To disconnect, remove plug from outlet.



10. Risk of electric shock - Connect this furnishing to a properly grounded outlet only. See Grounding Instructions.



11. Weight Limit: (patient and accessories) 625 lbs.



12. Be certain that the table is completely lowered without any tilt being present prior to discharging an ambulatory patient. The patient may lose balance and fall.



13. Electrical Shock Hazard. The power supply/control module is located below the base of the table. No user serviceable parts are inside. Refer servicing to qualified personnel. Unplug wall connector prior to contact with any cables connected to the power supply.



14. Keep fingers and hands away from underside of table top pinch points.

Celesta Deluxe™
- Model P5CA

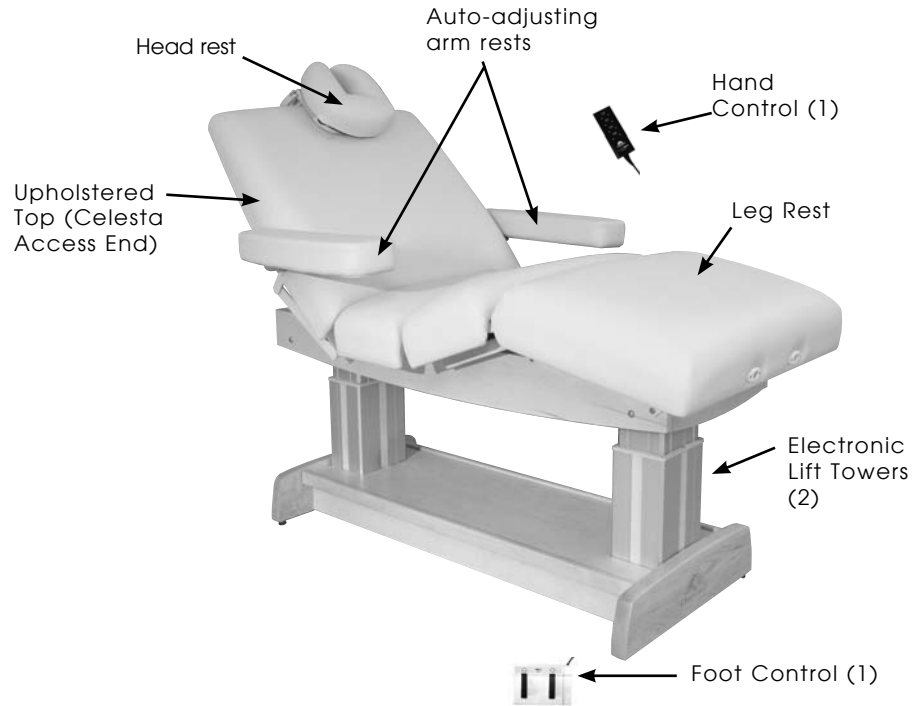


Table Specs* :

- Lifting Capacity: 625 lbs. (284 kg)
- Heights: 24"-35" (58 - 86 cm)
- Widths: 27", 29", 31" or 33"
(69, 74, 79, or 84 cm)
- Length: 72" (183 cm)
- Top: Electric Access Design Top
- Padding: Plush, 4" Aero•Cel™ or 6" Aero•Cel™
- Upholstery: TerraTouch™, UltraTouch™

* Published weight and height may differ slightly due to options chosen, natural material variations and padding density.
For electrical specs, see Specifications Chart

Celesta Lift™ - Model P5CT

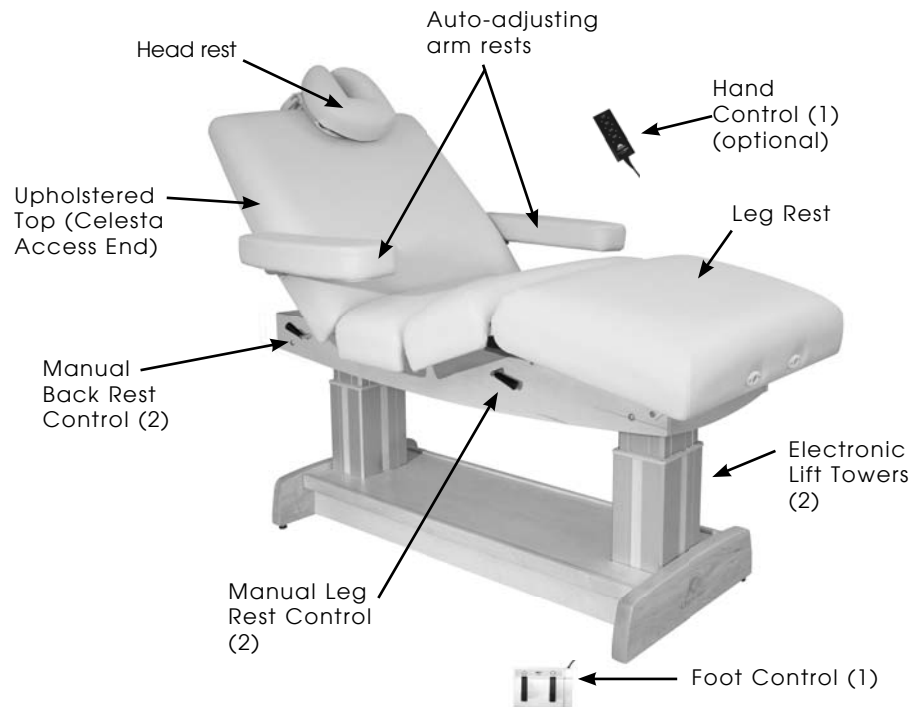


Table Specs* :

- Lifting Capacity: 625 lbs. (284 kg)
- Heights: 24"-35" (58 - 86 cm)
- Widths: 29" or 31" (74 or 79 cm)
- Length: 72" (183 cm)
- Top: Manually Controlled Access Design Top
- Padding: Plush, 4" Aero•CeI™ or 6" Aero•CeI™
- Upholstery: TerraTouch™, UltraTouch™

* Published weight and height may differ slightly due to options chosen, natural material variations and padding density.

For electrical specs see Specifications Chart

Celesta PerformaLift™ - Model P5

AVAILABLE WITH:

- RECTANGULAR FLAT TOP
- ROUND CORNER FLAT TOP
- BACKREST TOP
- FLEX TOP (SHOWN HERE)

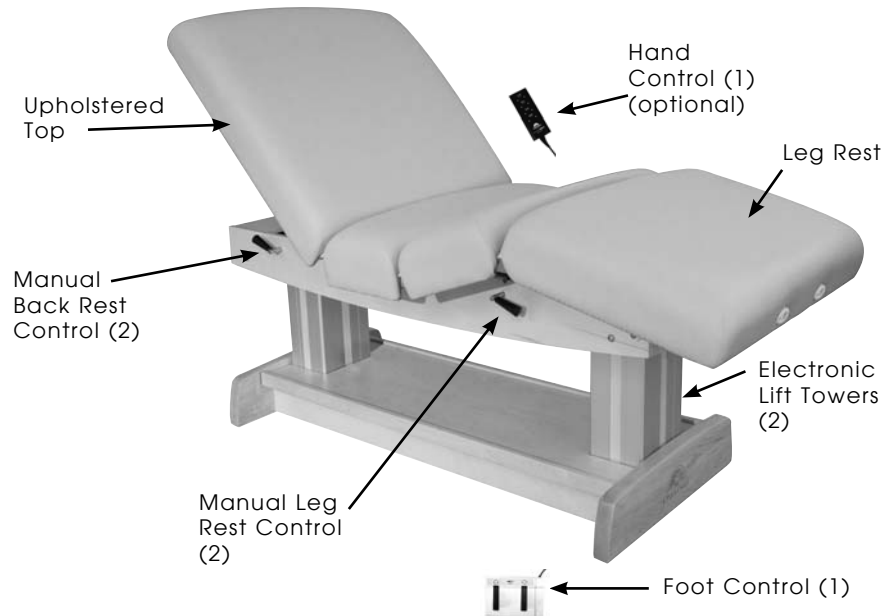


Table Specs* :

- Lifting Capacity: 625 lbs. (284 kg)
- Heights: 24"-35" (58 - 86 cm)
- Widths: Dependent on top
27", 29", 31" 33"
(69, 74, 79, or 84 cm)
- Length: Dependent on top
72", 73", 75"
(185, 186 or 190 cm)
- Tops: Rectangular, Rounded Corner, Back Rest & Flextop
- Padding: Plush (standard), Semi-Firm, Firm, 4" Aero•Cel™ or 6" Aero•Cel™
- Upholstery: TerraTouch™ (standard), Ultratouch

* Published weight and height may differ slightly due to options chosen, natural material variations and padding density.

For electrical specs, see Specifications Chart

INSTALLATION

The Celesta and PerformaLift tables come completely assembled and ready to use. Plug the cord into a functioning outlet that is rated for the table.


Arrange the power cord and control cords so that they will not create a tripping hazard and where the controllers are located to your liking and are conveniently accessible.

LEVELING INSTRUCTIONS:


If Table Will Not Sit Flat on the Floor:

The Celesta and Performance Lift Tables are equipped with Leveler Feet that adjust to level the table on uneven floors. To adjust the Leveler Feet to suit common floor differences you will need to use a 1/2" open-end wrench.

Leveler Foot

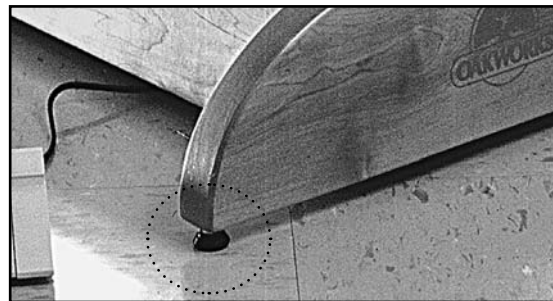


1/2" Open-end Wrench

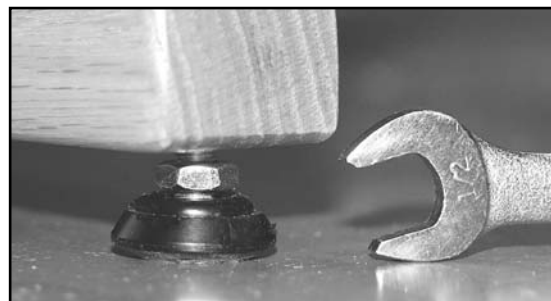


To Adjust the Leveler Feet:

1. Place the table in the location where you plan to use it.
2. Rock the table to determine if it is level with the floor. If the table rocks back and forth at all, the foot that clears the floor when you rock the table will need adjustment.
3. Adjust the Leveler Foot by first loosening the locking nut found just above the rubber Leveler Foot with the open-end wrench.

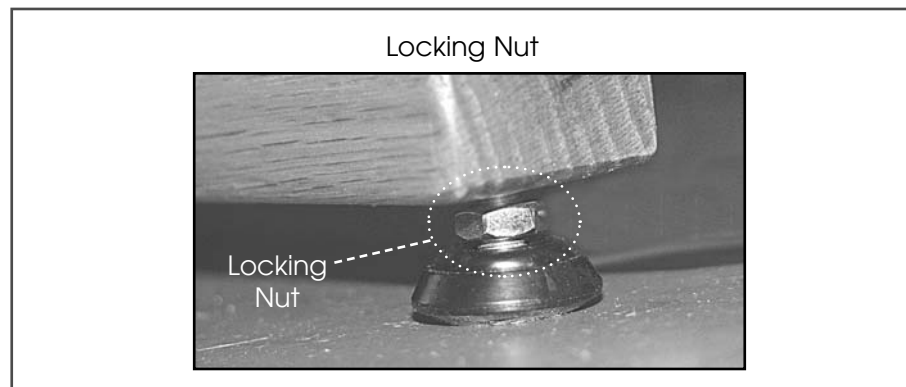

4. Lower the adjustable rubber Leveler Foot until it meets the floor snugly. Tighten the locking nut against the wooden leg of the table with the open-end wrench to fix the Leveler Foot in position.

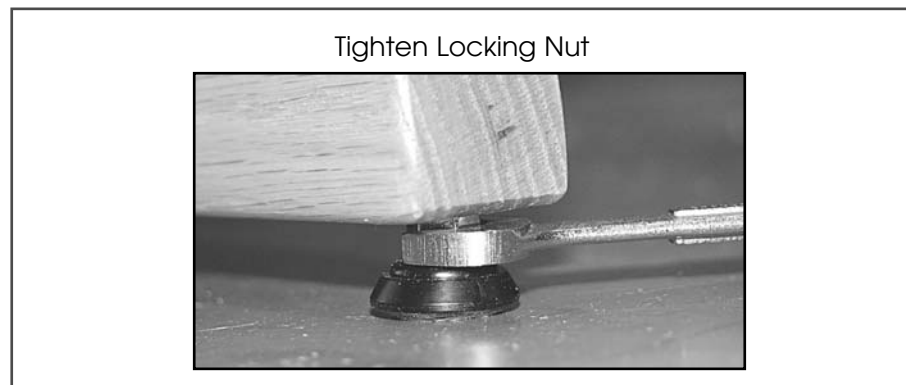

5. Rock the table again to be sure adjustments have stabilized the table. If necessary make any additional adjustments in the same way.

DIRECTIONS FOR USE

The Oakworks Lift Tables offer a variety of positioning capabilities when client comfort is critical. They are especially effective for professional protocols that require specific positioning of the client and provide superior ergonomics for the comfort level of the professional therapist as well.

The OAKWORKS® Celesta Deluxe, Lift and PerformaLift come complete with a foot control and hand control to operate the height and angle of the table. The back rest and leg rest features of the top are manually adjusted on the Celesta Lift and PerformaLift. The Celesta Deluxe features a top that is operated electrically by a hand control.

Celesta® Deluxe


Celesta® Lift

Celesta® PerformaLift

FOOT CONTROL OPERATIONS

Foot Control:

Operate the foot control as shown below to raise or lower the height of the table. When operating the table's controls, be sure to observe all cautions and warnings as given in the manual.

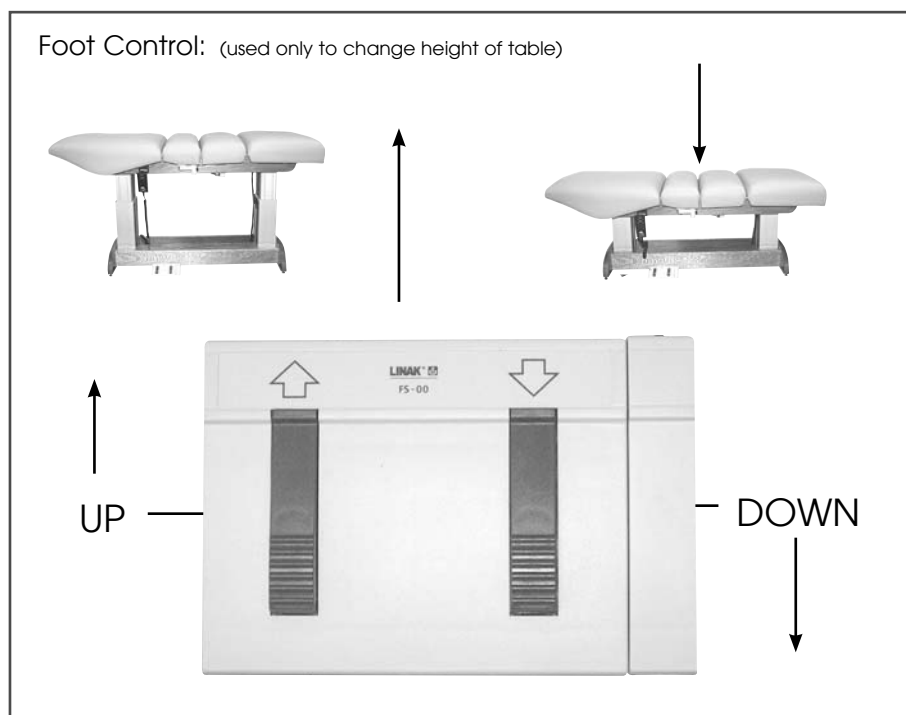

CAUTION

When operating the table controls be sure to observe all cautions and warnings given in the manual to prevent injury to both operator and client.

DIRECTIONS FOR USE

Adjusting Height and Tilt Angle - Hand Control - Celesta Lift, Celesta PerformaLift


Hand Control:

The Hand Control operates the tilt angle and the height of the table with the touch of a button. Follow the directions below to access these functions.

Celesta® Lift

Celesta® PerformaLift

HAND CONTROL OPERATIONS

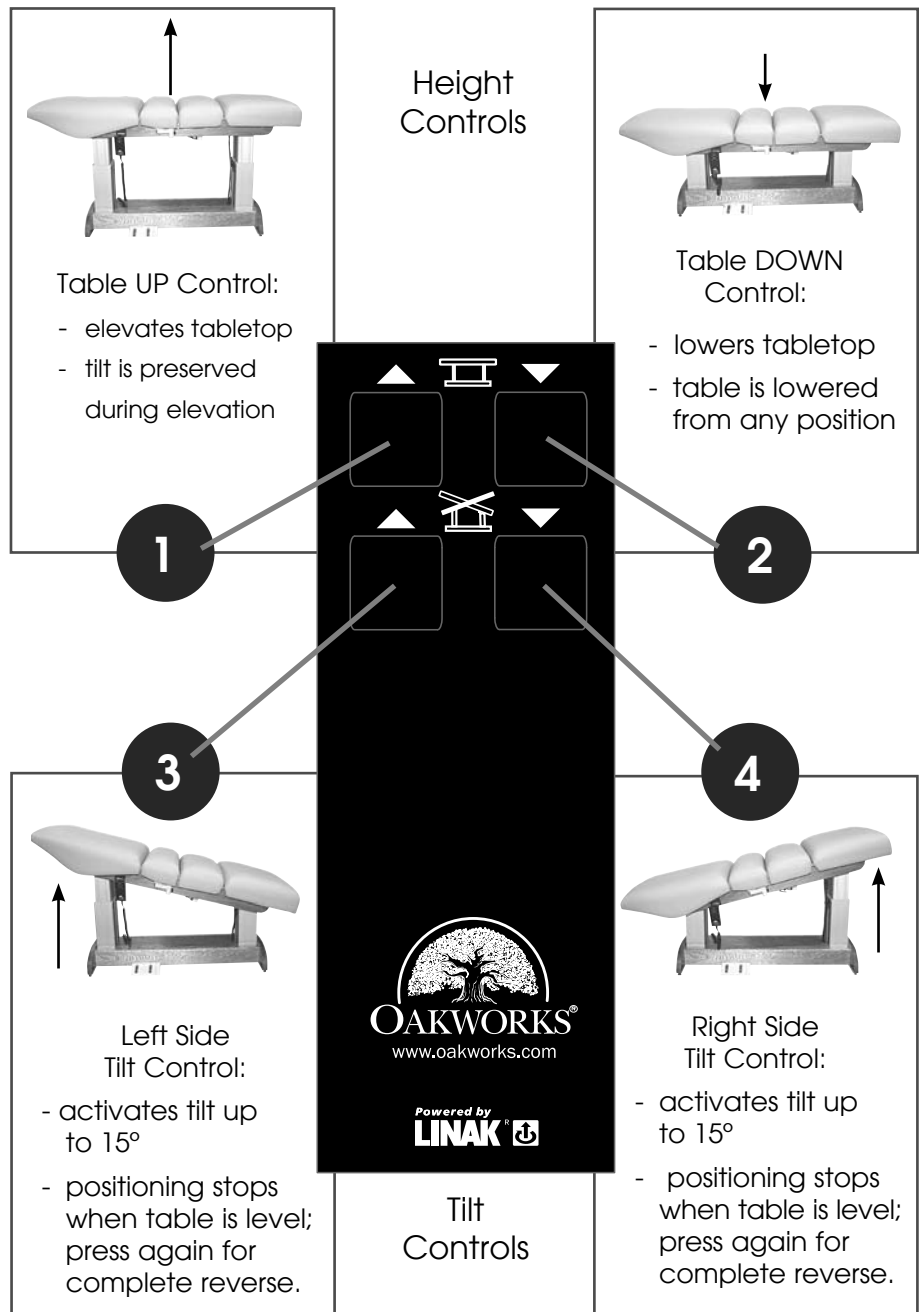

CAUTION

Do not sit on the elevated Back Rest or Leg Rest.

CAUTION

Do not leave clients unattended when table is tilted near its limits.

Celesta® Deluxe

HAND CONTROL OPERATIONS

 **CAUTION**


Do not sit on the elevated Back Rest or Leg Rest.

 **CAUTION**

Do not sit on the elevated Back Rest or Leg Rest.


 **CAUTION**

Do not leave clients unattended when table is tilted near its limits.

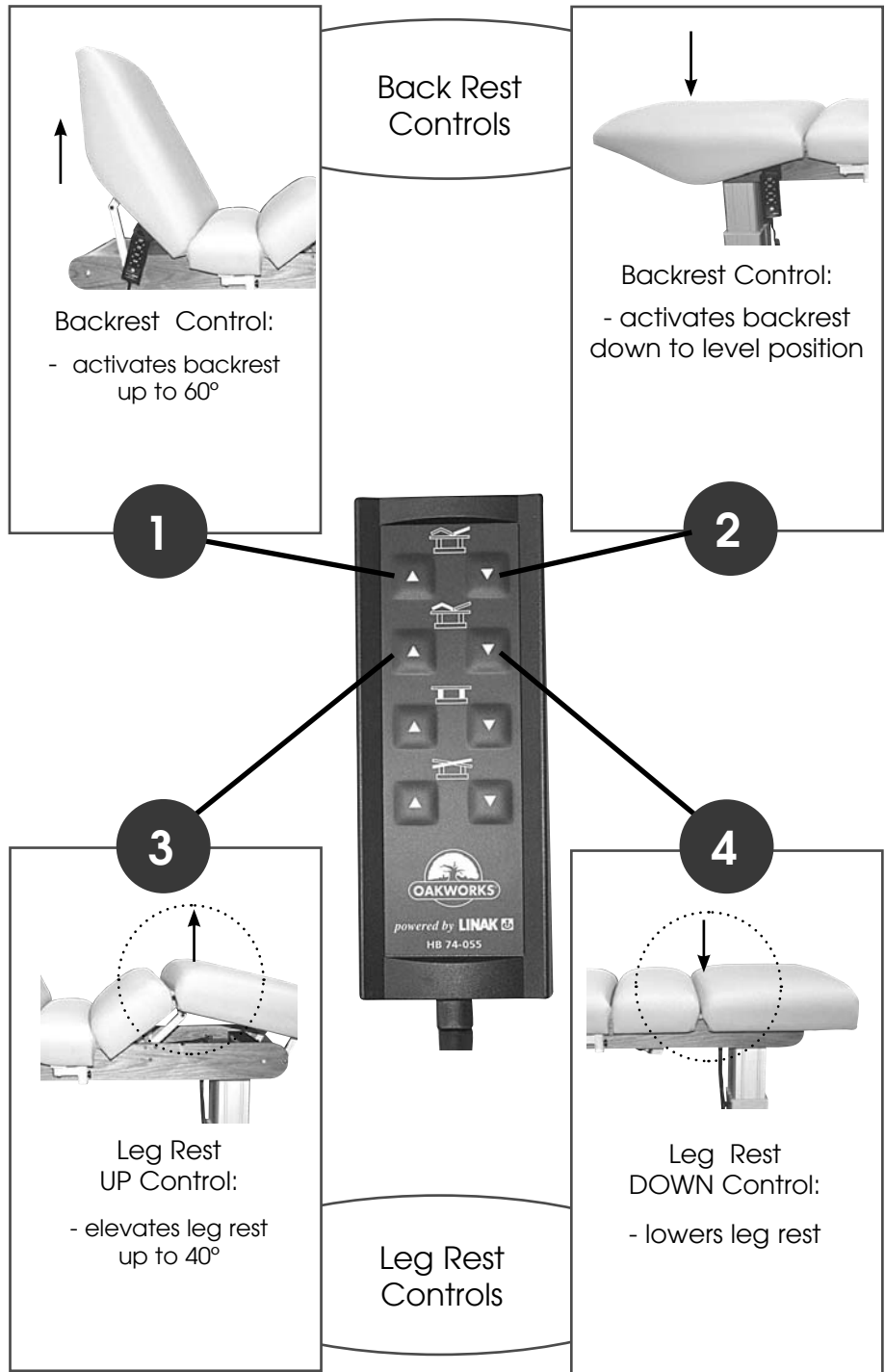

Celesta® Deluxe

HAND CONTROL OPERATIONS

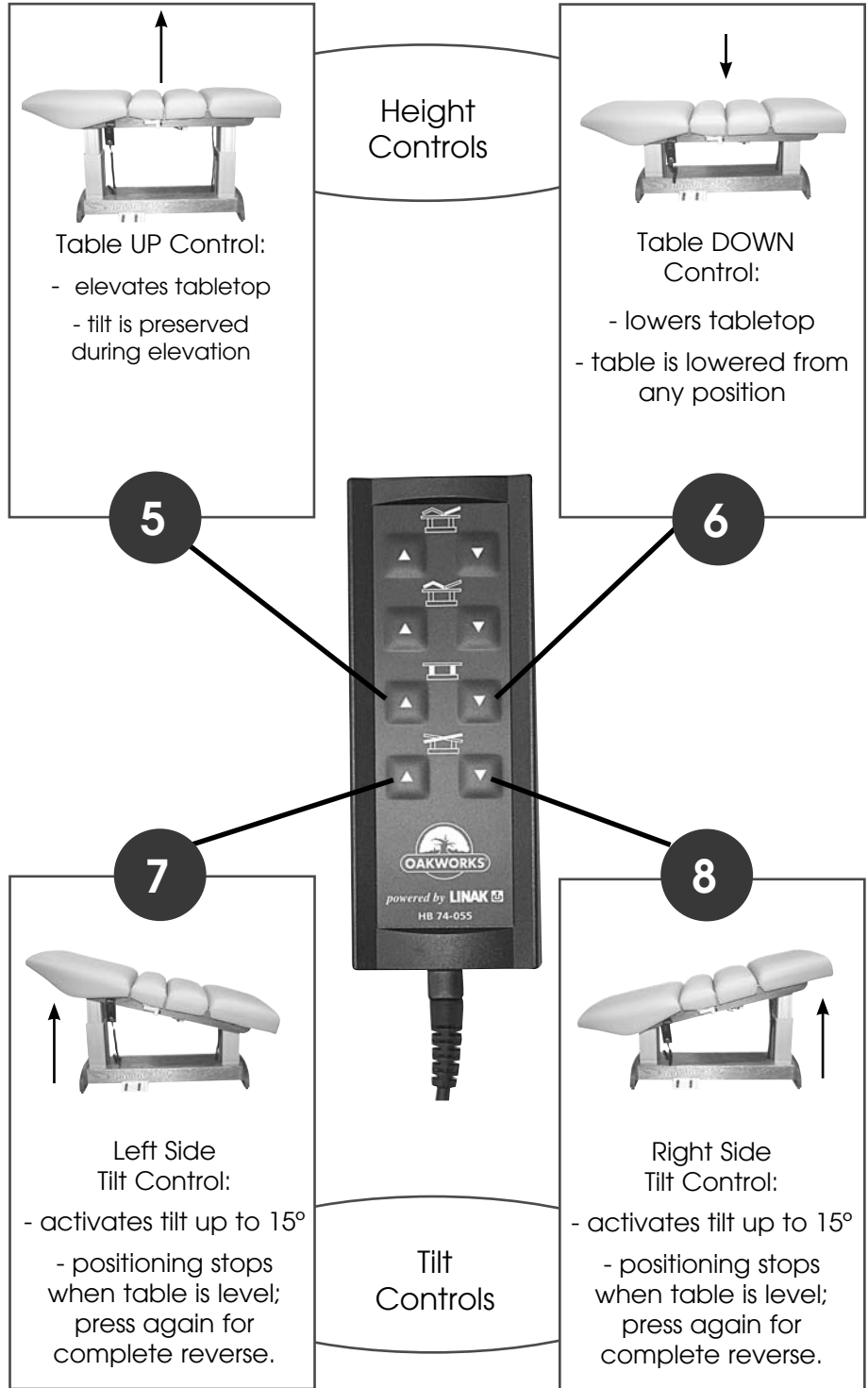

 **CAUTION**

Do not leave clients unattended when table is tilted near its limits.

Celesta® Lift

Celesta® PerformaLift (backrest or flex top)


OPERATING THE MANUAL TOP

To Raise the Back Rest:

Raise the backrest by pulling it up from the head end of the table to the proper position for the procedure. It is recommended that the client takes some of their weight off the backrest while you lift.

 **CAUTION**

Do not sit on the elevated Back Rest or Leg Rest.

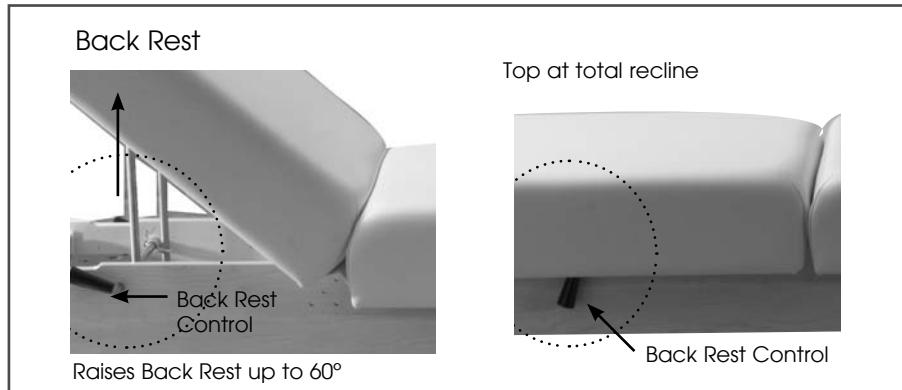

To Lower the Back Rest:

Support the backrest and weight of the client with one hand. Push the locking handle towards the center of the table slowly to loosen the locking mechanism and lower the backrest to the desired position. Release the handle and the backrest will stay in place. It is recommended that the client lifts themselves off the backrest to take some of the weight off the backrest while you move it into position.

 **CAUTION**


Do not release backrest without supporting the backrest and client

To Raise the Leg Rest:

Raise the leg rest by pulling it up from the lower center end of the table to the proper position for the procedure. It is recommended that the client takes some of their weight off the leg rest as you lift.

Celesta® Lift

Celesta® PerformaLift (flex top)

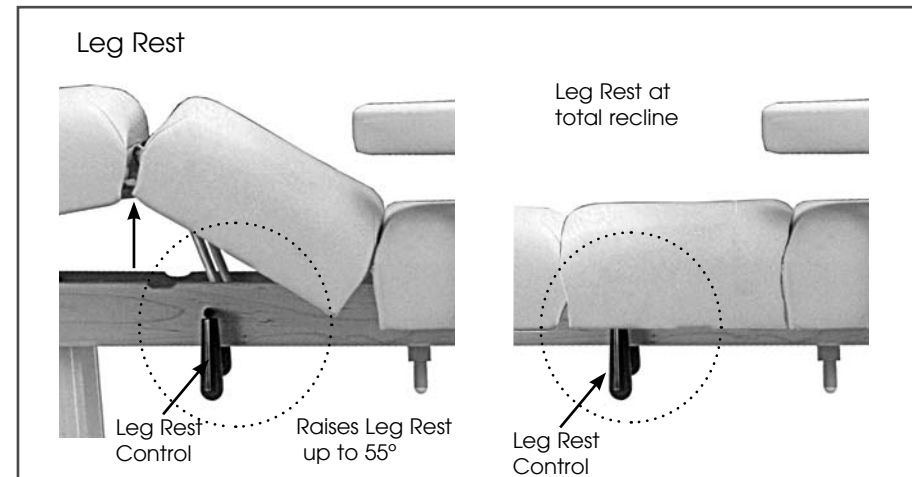

To Lower the Leg Rest:

Support the leg rest and weight of the client with one hand. Push the locking handle towards the center of the table slowly to loosen the locking mechanism and lower the leg rest to the desired position. Release the handle and the leg rest will stay in place. It is recommended that the client lifts the weight off the leg rest while you move it into position.

 **CAUTION**

Do not release leg rest without supporting the leg rest and client




Be sure to read all cautions, warnings and instructions given in the manual to prevent injury to both operator and client.

GROUNDING

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electrical current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances. See U.S. sample below.

This product is manufactured for use on a nominal 120-volt circuit for the United States and a nominal 220-volt circuit for use in Europe. The plug has a grounding pin in a pattern that varies by country as shown in the Product Identification section of this manual. Make sure that the product is connected to an outlet having the same configuration as the plug.

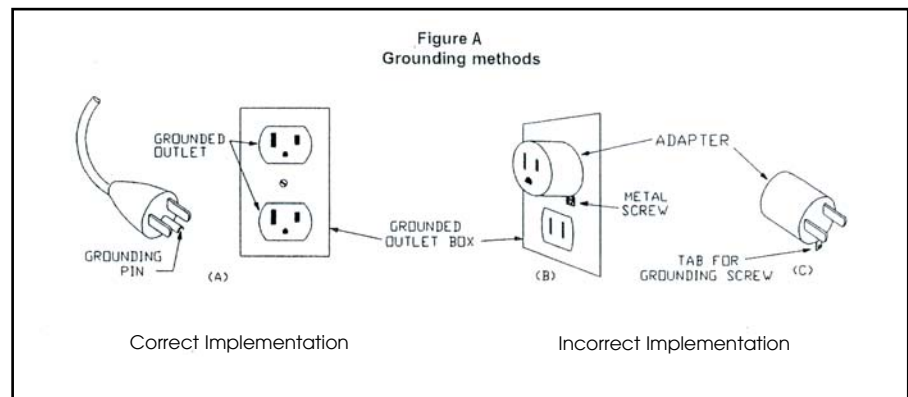

CAUTION

DANGER-Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or service man if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product - if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

To reduce the risk of electric shock, this furnishing has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

- Side Arm Rests

**CAUTION**

Do not sit on the Arm Rests.

Do not place excess weight on the Side Arm Rests. Do not use them in assisting the client on and off the table.

Either Arm Rest can be repositioned to allow a client to get on and off the table easily.

When making adjustments the arm rests make certain that the client's arms are away from the working mechanism.



Be sure to read all cautions, warnings and instructions given in the manual to prevent injury to both operator and client.

SIDE ARM RESTS

Though the arm rests adjust automatically to remain parallel to the floor in any position, there will still be times when you will need to move an arm rest out of the way or remove them altogether. We recommend that you raise the arm rest out of the way to allow a client to get on or off the table. They are designed to be strong and will take a reasonable amount of weight but they are not designed to be used to sustain a person's weight when getting on or off the table.

To Adjust the Arm Rest:

Before asking a client to mount or dismount from the table we recommend that you move the arm rest out of the way. The arm rests are easy to move and manually adjust. First position the tabletop with the backrest in the upright position and all other section flat. Take hold of the arm rest firmly and raise the arm rest up and out of the way with a steady swing until the arm rest is in the highest upright position (see images below). Hold it in place until your client has mounted (or dismounted) and then actively move it back into the resting position where it will proceed to move automatically with the table top.

Adjusting the Side Arm Rest



The arm rest is now set and will adjust automatically as you position the client for whatever procedure is needed.

There will be times when removal of the arm rests will be needed. This feature is helpful when doing massage or helping someone get on or off the table. This can be done easily and storage is conveniently located in the under-table cabinet. They are then at hand when you are ready to re-install.

To Remove the Arm Rest:

The arm rest frame has a notched connection to the interior of the table top which allows it to move automatically when the table top changes position. (cont. on next page)

DIRECTIONS FOR USE

side arm rests

SIDE ARM RESTS: (CONT.)

• Side Arm Rests (cont.)

IMPORTANT



CAUTION

Do not sit on the Arm Rests.

Do not place excess weight on the Side Arm Rests. Do not use them in assisting the client on and off the table.

Either Arm Rest can be repositioned to allow a client to get on and off the table easily.

When making adjustments the arm rests make certain that The client's arms are away from the working mechanism.



CAUTION

Be sure the arm rest is fully inserted before lowering.

There will be times when removal of the arm rests will be needed. This feature is helpful when doing massage or helping someone get on or off the table.

To Remove the Arm Rest:

To remove one or both arm rests just take hold of the base of the arm rest at the metal frame and somewhere on the cushion. Holding securely, rotate the arm rest while at the same time pulling slightly away from the table. You will feel the notched connection slide out as you rotate the arm rest. When you feel this you can then slide the entire arm rest out of the connection hole very easily. Store it safely until ready for installation.

Re-attach the armrest by sliding the arm rest frame into the hole on the backrest. Push it in securely and allow the arm rest to rest in the parallel position. This resets the arm rest in the proper position for the automatic adjustment feature to be activated.

Removing the Side Arm Rests



MANICURE ARM RESTS:

The manicure arm rest installation follows the same standard installation and removal instructions as above.

- A. hand soak bowl installation
- B. hand soak bowl removal
- C. Manicure surface installation



A



B



C


- Lowered Arm Rest Shelf
- Lowered Arm Rest Shelf with Aromatherapy

For a totally comfortable and relaxed client, add a Lowered Arm Rest Shelf to your lift table. It's covered in the same luxurious plush padding as the table itself. The Lowered Arm Rest Shelf can be removed when not in use and stored away. It can also act as a table extender, when used in place of the face rest or at the foot of the table. For complete use & care instructions see product instructions "Arm rest shelf instructions"

 **CAUTION**

Do not sit on or put excessive weight or pressure on lowered arm shelf.

Lowered Arm Rest Shelf with Aromatherapy, shown here with QuickLock™ Face Rest

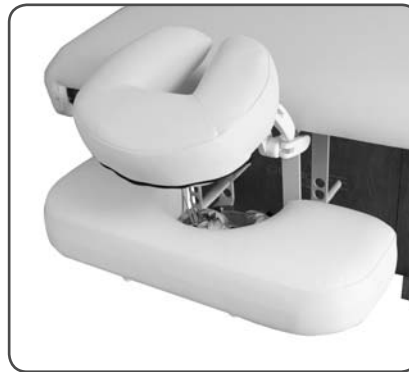

Lowered Arm Rest Shelf, shown here used as a table extender



- QuickLock™ Face Rest*

IMPORTANT

 **SAFETY NOTE:**

Do Not extend the dowels of the Face Rest further than 3" from the table outlets to ensure a stable connection.

Double articulating action provides nearly limitless positioning options with the QuickLock™ Face Rest. The adjustable pad takes pressure off sensitive facial areas while providing superb support. The QuickLock™ Face Rest can be used in both prone and supine positioning and folds out of the way when not in use. Adjustments are made easily by loosening the cam locks, adjusting for height and angle and then locking the cam locks into that position.

To remove the QuickLock™ Face Rest, just pull the frame from the outlets in the end of the table until it comes free. When replacing the QuickLock™ Face Rest, just push the frame back into the outlets being sure to keep the dowels within 3" of the table to ensure a stable connection. For complete use & care instructions see product instructions "QuickLock Face Rest instructions"

 **CAUTION**

Do not put excessive weight or pressure on the face rest.

* U.S. Patent #5,177,823

QuickLock™ Face Rest



Important Safety Note: Maximum Distance from edge of Table = 3"



TROUBLESHOOTING

IMPORTANT



CAUTION

The table weighs as much as 300 lbs. DO NOT lift or move the unit without using proper safety procedures or equipment. DO NOT turn the table on its side for any reason. To do so could cause damage to the base and will void the warranty. There are no serviceable items that need this action.



Be sure to read all cautions, warnings and instructions given in the manual to prevent injury to both operator and client.

THERMAL LIMITING SWITCH

The Celesta Lift, Celesta Deluxe and Celesta PerformaLift tables have two lifting columns that raise and lower the table. In addition, the Celesta Deluxe has two actuators that allow the client to be positioned for both client and therapist comfort. Each of these devices has a duty cycle of approximately 10%, meaning 2 minutes of operation within a 20 minute period. These devices have a thermal limiting switch to prevent overheating. As the thermal limiting switch is engaged, the motion will slowly come to a stop. Since there are 4 independent motion controls, the table may be able to go up but one of the positioning motions may not work. Allow the device to rest and it will automatically reset itself.

IF THE TABLE WILL NOT CHANGE POSITION

- Check the outlet to be sure that it has power and that the power cable is plugged in. Try each foot control and hand control to see if any work. If any work, then possibly there is a loose connection or a faulty controller.
- Unplug the power cable from the wall outlet.
- Make sure that all male and female connectors are firmly pushed into each other. Even a slight misconnection can cause a possible malfunction.
- Plug the unit back into the wall and check the controls again. If none work, then contact customer service.

QUESTIONS TO ASK BEFORE CALLING CUSTOMER SERVICE

- Is the unit plugged in?
- Does the outlet have power?
- Check all connections of the control box, is everything seated securely?
- Does the height of the table work when you press on the foot pedal?
- Does the top work when you use the hand control?

WARRANTY INFORMATION

View complete warranty details at www.oakworks.com

CARE & CLEANING

HOW TO CARE FOR YOUR UPHOLSTERY

In order to maintain the quality and ensure the life of the table, here are some preventative steps that can be taken to protect the upholstery on the chair and accessories.

- Upholstery

- Protect your table and accessories with sheets and fleeces. We offer both fitted sheets and thick fleeces that protect your upholstery fabric against oil stains while offering comfortable 100% cotton flannel softness to your clients. You can also preserve your face rest and bolsters with 100% cotton flannel covers and fleeces.
- Clean your upholstered items only when needed with a mild detergent or Green Windex. A diluted 1/100 bleach solution for disinfecting can be used. Dry the table immediately after cleaning to remove excess cleaning solution. Using strong cleaners, such as alcohol, acetone, higher concentrations of bleach or other products that contain high concentrations of these substances, can shorten the life of your fabric, or discolor it. *Note: Damage caused by them will not be covered under the warranty.*
- Avoid extremes in heat. Very cold temperatures can make your fabric prone to cracking. Very high temperatures can cause the fabric to stretch permanently.
- Avoid exposing your products to direct sunlight for extended periods of time. This can lead to fading and cracking of the fabric.
- Use an anti-slip pad to prevent oil from reaching the upholstery and to keep your sheets from slipping.

GENERAL CARE OF EQUIPMENT COMPONENTS

- Base & Controls






- Use only a mild solution of 4:1 diluted non-alcohol cleaner such as 409®, Fantastik® or some other non-abrasive cleaner to wipe down the metal surfaces of the chair. Apply with a damp (not wet) lint-free cloth. Be sure excess liquid does not drip onto, or into, any of the electrical controls or mechanisms. Be sure that the unit is unplugged before cleaning these areas. Wipe these areas carefully with a lint free cloth.
- The electronic powering mechanism of your table requires no general maintenance or lubrication.
- The wooden portions of the equipment can be polished with any polish approved for wooden surfaces. Spray, or pour, the polish onto a soft cloth and be careful not to get it onto other surfaces.

PRODUCT IDENTIFICATION

- Power Lead Plugs

POWER LEAD PLUGS

OAKWORKS Part #.:	Voltage (AC)	Power Cord Plug
5097-11	120v 60 Hz	3 prong grounded hospital grade; North America
5097-3	220v 50 Hz	Continental Plug; Europe
5097-4	220v 50 Hz	Swiss plug; Switzerland
5097-2	220v 50 Hz	British plug; United Kingdom
N/A This cord is not detachable.	100v 50/60 Hz	2 prong polarized plug; Japan & Korea

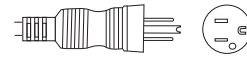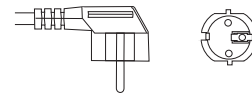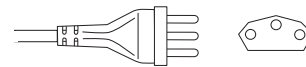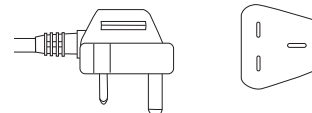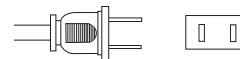SPECIFICATIONS

Model No.	All Models
-----------	------------

ELECTRICAL SPECIFICATIONS




Designed For:	North America	Europe	Japan/Korea
Input Voltage	120 VAC	220 VAC	100 VAC
Input Connection	15 amps	10 amps	20 amps
Input Power	5.8 amps	3.0 amps	6.8 amps
Input Frequency	60 Hz	50 Hz	50/60 Hz
Voltage Output	24 VDC	24 VDC	24 VDC
Power Consumption	1.5 amps	1.5 amps	1.5 amps
Maximum Momentary Power Consumption	9.0 amps	9.0 amps	9.0 amps
Electric Shock Protection	Class I Equipment	Class I Equipment	Class I Equipment
Tabletop Applied Part	 Type B Applied Part	 Type B Applied Part	 Type B Applied Part
Table Top IEC 529 Rating	IPX0	IPX0	IPX0
Mode of Operation	Continuous Operation @ 10% Duty Cycle	Continuous Operation @ 10% Duty Cycle	Continuous Operation @ 10% Duty Cycle

TABLE SPECIFICATIONS

Designed For:	North America	Europe	Japan/Korea
Length:			
Deluxe & Lift	72"	182 cm	182 cm
Rectangular	73" or 75"	185 cm or 190 cm	185 cm or 190 cm
Rounded Corner	73" or 75"	185 cm or 190 cm	185 cm or 190 cm
Back Rest Top	73" or 75"	185 cm or 190 cm	185 cm or 190 cm
Flextop	72"	182 cm	182 cm
Width:			
Celesta Deluxe	27", 29", 31", 33"	69, 74, 79, or 84 cm	69, 74, 79, or 84 cm
Celesta Lift	29", 31"	74 cm or 79 cm	74 cm or 79 cm
Rectangular	27", 29", 31", 33"	69, 74, 79, or 84 cm	69, 74, 79, or 84 cm
Rounded Corner	27", 29", 31", 33"	69, 74, 79, or 84 cm	69, 74, 79, or 84 cm
Back Rest Top	27", 29", 31", 33"	69, 74, 79, or 84 cm	69, 74, 79, or 84 cm
Flextop	29", 31"	74 cm - 79 cm	74 cm - 79 cm
Height Range	24" - 35"	58 cm - 86 cm	58 cm - 86 cm
Weight	~ 300 lbs.	~ 136 kg	~ 136 kg
Shipping Weight	~ 380 lbs.	~ 173 kg	~ 173 kg
Lifting Capacity	625 lbs.	283 kg	283 kg

Storage & Transport	 Temperature: -10° C - 60° C	Humidity: 60% relative humidity During transport, DO NOT stack containers. 	Pressure: no limitations known
---------------------	---	---	--------------------------------

INSTRUCTION MANUAL

OAKWORKS® Lift Tables

CONTACT INFORMATION:

OAKWORKS® Inc.

923 East Wellspring Road
New Freedom, PA 17349

Toll Free (USA only): 800-916-4613
Phone: 717-235-6807

FAX: 717-235-6798

www.oakworks.com

European Compliance for CE Mark Information

Emergo Europe

Molenstraat 15
The Hague, 2513 BH Netherlands

Tel: +31 70 3458570

Fax: +31 70 3467299

E-mail: info@emergogroup.com

Manual Part Number MMMNST0003

Revision date: 10.26.09

2nd Edition, October 2009

1st Edition, September 2003

English, Printed in USA

