

PowerLine™ Treatment Table Optional Accessories & Table Care

Attaching the Table Extender or Paper Roll Holder:

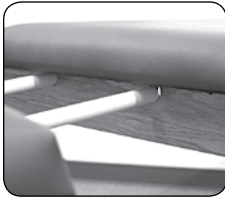


Fig. 7

1. The table is equipped with two outlet holes at either end to allow the use of a Table Extender and a Paper Roll Holder. Unfold the aluminum dowels of the extender and insert them into the outlet holes at either end of the table. (Fig. 7 & Fig. 8) The Paper Roll Holder has dowels to insert into the outlet holes as well and will be held securely for ease of use. (Fig. 8)

CAUTION: DO NOT sit, lean or use the Extender or Paper Roll Holder in mounting or dismounting from the table.

Attaching & Using the QuickLock™ Face Rest:

1. Insert the ends of the Face Rest frame into the outlet holes at either end of the table. (Fig. 10)
2. Starting with the face rest in a flat position, have your client lie face down on the face rest. Ask your client to support the face rest with two hands by holding onto it at the velcro on the sides of the U-shaped tube after you have unlocked the cam locks. (Fig. 11)
3. Have your client pull the face rest towards him/her, raising the chin and lowering the forehead until he/she is comfortable on the pad; cheeks fully supported and neck comfortably flexed. (Fig. 12)
4. When the face rest is in a position that meets your treatment requirements and your client's optimal comfort, lock the cam locks. (Fig. 13)

CAUTION: DO NOT extend the dowels of the Face Rest further than 3" from the table portals to ensure a stable connection. (Fig. 14) The maximum weight capacity for the QuickLock™ Face Rest is 25 lbs. DO NOT sit, lean or use the face rest in mounting or dismounting from the table

Cleaning & Maintenance of the PowerLine table:

- Clean your upholstered items only when needed with a mild detergent or Green Windex. A diluted 1/100 bleach solution for disinfecting can be used. Dry the table immediately after cleaning to remove excess cleaning solution. Using strong cleaners, such as alcohol, acetone, higher concentrations of bleach or other products that contain high concentrations of these substances, can shorten the life of your fabric, or discolor it. *Note: Damage caused by these cleansers will not be covered under the warranty.*
- Avoid extremes in heat. Very cold temperatures can make your fabric prone to cracking. Very high temperatures can cause the fabric to stretch permanently.
- Avoid exposing your products to direct sunlight for extended periods of time. This can lead to fading and cracking of the fabric.
- The wooden cabinetry of the equipment can be polished with any polish approved for wooden surfaces. Spray, or pour, the polish onto a soft cloth and be careful not to get it onto other surfaces.

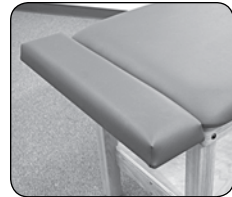


Fig. 8



Fig. 9

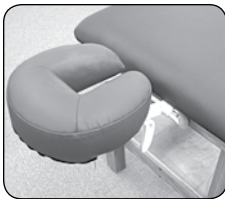


Fig. 10



Fig. 11



Fig. 12



Fig. 13

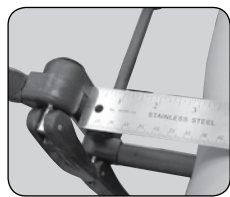


Fig. 14

PowerLine™
by OAKWORKS®

The POWER of STRENGTH

Congratulations!

You have just bought the best treatment table built with bariatric rated features designed to make it last under the most demanding circumstances.



H-Brace

Removable Shelf

Backrest Top

PowerLine™ by OAKWORKS®

H-Brace and Shelf

Supplied Parts:

- | | | |
|----------------------------------|----------------------|---|
| 1 - Stretcher Beam | 4 - Long Large Bolts | 1 - Large Allen Wrench |
| 12 - Flat Washers (shiny finish) | 4 - Short Bolts | 1 - Small Allen Wrench |
| 4 - Disc Washers (dull finish) | 8 - Velcro™ Strips | Options: Shelf, Table Extender,
QuickLock™ Face Rest |
| 4 - Lock Nuts | 1 - Open End Wrench | |

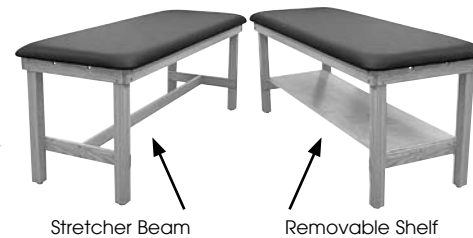


Fig. 1

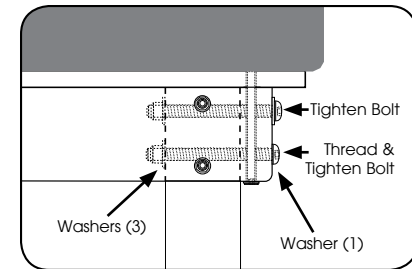


Fig. 2

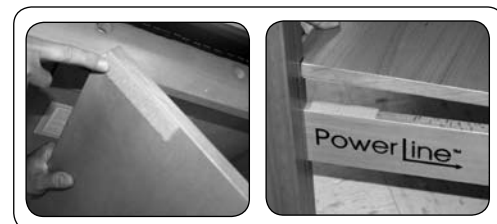


Fig. 3

Assembly Instructions

1. Remove the shelf from the box (if ordered). Remove the table from the box and place it down on the floor. Remove all hardware and the stretcher beam.
2. Unfold the legs to a vertical position and install the Long Large Bolts, Washers and Nuts, as shown, on the top of each leg. When placing washers inside of the leg, sandwich 1 Disc Washer between 2 Flat Washers; outside of the leg, use 1 Flat Washer. Using the supplied tools, tighten every bolt -- 8 total -- as tight as possible. (Fig. 1)
3. Install the Short Bolts on either end of the stretcher beam and tighten the bolts on both ends. Make sure the stretcher beam is flush with the adjoining rails while tightening bolts. (Fig. 2)
4. Turn your table right side up. Get up on the table and rock back and forth to check that all bolts are tight. The table should be very stiff. If not, check and tighten the bolts as necessary with the supplied tools.

NOTE: Retighten all bolts after 2 weeks; the wood will naturally experience slight shrinkage, which can cause the bolts to loosen.

Shelf Instructions

If you have ordered a shelf, drop the shelf into place with the finished side up. The shelf has grooves at both ends that enable it to be held firmly in place by the legs. If necessary, the shelf can be further stabilized by installing the enclosed Velcro™ strips. Place strips on each corner of the shelf's bottom side, and on each end of the rails. Make sure not to mismatch Velcro™ strips; place all "hook" or all "loop" pieces on either the rail or shelf. (Fig. 3)

PowerLine™ by OAKWORKS®

Backrest Top

Supplied Parts:

- | | | |
|----------------------------------|----------------------|---|
| 1 - Stretcher Beam | 6 - Long Large Bolts | 1 - Large Allen Wrench |
| 18 - Flat Washers (shiny finish) | 4 - Short Bolts | 1 - Small Allen Wrench |
| 6 - Disc Washers (dull finish) | 8 - Velcro™ Strips | Options: Shelf, Table Extender,
QuickLock™ Face Rest |
| 6 - Lock Nuts | 1 - Open End Wrench | |



Backrest Top

Assembly Instructions

1. Remove the table and leg assembly from the box and place it down on the floor. Remove all hardware and the stretcher beam. Remove the shelf from the box* (if ordered).
2. Unfold the legs to a vertical position on the foot end of the table. Install the Long Large Bolts, Washers and Nuts, as shown, on the top of each leg. (see Fig. 1) When placing washers inside of the leg, sandwich 1 Disc Washer between 2 Flat Washers; outside of the leg, use 1 Flat Washer. Using the supplied tools, tighten every bolt -- 4 total -- as tight as possible.

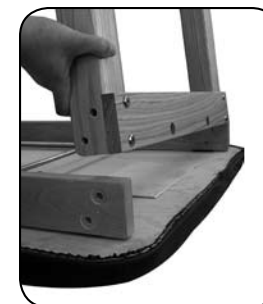


Fig. 4

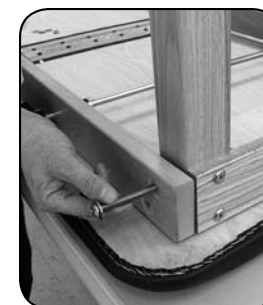


Fig. 5

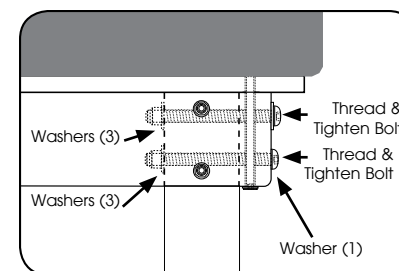


Fig. 6

You will need to install the legs on the head end of the table. Line up the leg assembly as shown in Fig. 4. Install the Long Large Bolts, Washers and Nuts, as shown in Fig 5, on the top of each leg. When placing washers inside of the leg, sandwich 1 Disc Washer between 2 Flat Washers; outside of the leg, use 1 Flat Washer. Using the supplied tools, tighten every bolt -- 4 total -- as tight as possible. (see Fig. 6)

3. Install the Short Bolts on either end of the stretcher beam and tighten the bolts on both ends. Make sure the stretcher beam is flush with the adjoining rails while tightening bolts. (Fig. 2)
4. Turn your table right side up. Get up on the table and rock back and forth to check that all bolts are tight. The table should be very stiff. If not, check and tighten the bolts as necessary with the supplied tools.

NOTE: Retighten all bolts after 2 weeks; the wood will naturally experience slight shrinkage, which can cause the bolts to loosen.

* Refer to shelf instructions on previous page