

# Wellspring™, Advanta™, Alliance™ Metal, Storable Mat



Fig. 1

## To Set-up the Table:

- Place the folded table sitting on the rubber bumpers; with the heel of your hand, bump the UniLock™ tab between the face rest outlets and the table will open. (Fig. 1)
- Open the table part way, making sure the cables are still slack (Fig. 2)
- Press the “Y” brace against the table, then pull up on the legs one at a time. (Fig. 3) Do the same on the other end. Open the table until the leg braces and table top are completely extended and unwrap any cables that may be caught in the legs.



Fig. 2



Fig. 3

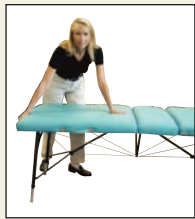


Fig. 4

## To Set the Table Upright:

- When the table is fully opened, tilt the table upright until all four feet are on the floor. Lift one end of the table, press down on the middle and tension the cables a little. (Fig. 4)

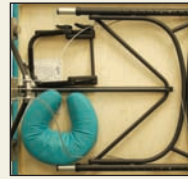


Fig. 5



Fig. 6

## To Close the Table:

- Turn the table on its side with the rubber bumpers down. If you own a face-rest, store it under the table (Fig. 5). Partially close the table top.
- Fold the legs into the table and make sure the cables are inside of the table. Push the two halves of your table together to lock. (Fig. 6).

*Note: Advanta leg extensions must be fully retracted to close the table.*

## To Change the Table Height:

- With no weight on the table, press the button on the leg and rotate the leg extension slightly to take the button out of alignment with the holes, but still within sight. Adjust all extensions to the desired height and turn until the button pops out of the aligned hole. (Fig. 6)



Fig. 6

NOTE: OAKWORKS® is not liable for injuries or damages sustained when table is not used in the manner prescribed.

## Backrest



- To open/adjust - Release velcro tab located under the center end of the table and pull up. Raise the center of the backrest, stopping at desired height
- Make sure that both sides of the tilting hardware are on the same setting and are both locked in place.
- To close, raise the backrest fully and lower to closed position. Re-attach velcro tab to hold backrest in closed position. Refer to warning on under side of table for precautions.

## CAUTION

Dynamic Load Rating:  
**550 lbs.**

**Made in the U.S.A.**  
*with U.S. & Imported Parts*

Inspected By: \_\_\_\_\_

Oakworks®  
*Original*

All portable tables:  
Unilock™  
Integra Hinge™  
CS System:

CE

U.S. Patent 5,009,170  
Patent Pending  
U.S. Patent 5,943,965  
U.S. Patent 6,192,809

OAKWORKS®, Inc.

923 East Wellspring Rd., New Freedom, PA 17349

Toll Free Phone: 800-558-8850  
Phone: 717-235-6807 FAX: 717-235-6798  
www.oakworks.com

  
**OAKWORKS®**  
Bodies Talk. We Listen.