

USER MANUAL: IMAGING AND PAIN MANAGEMENT ACCESSORY - LITHO/UROLOGY TABLE DRAPE



Fig. 1

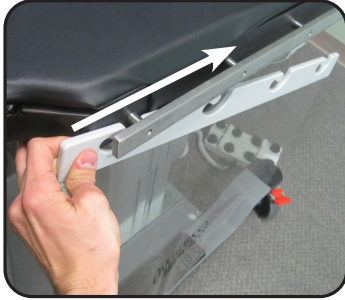


Fig. 2



Fig. 3

INSTRUCTIONS FOR INSTALLING THE DRAPE

1. Unfold the drape from its packaging.
2. Be sure the Oakworks Logo is facing outward before attaching (Fig. 1).
3. Slide one side of the drape attachment onto the T-Rail Supports on the table (Fig. 2).
4. Slide the other drape attachment onto the opposite side of the table's T-Rail supports (Fig. 3).

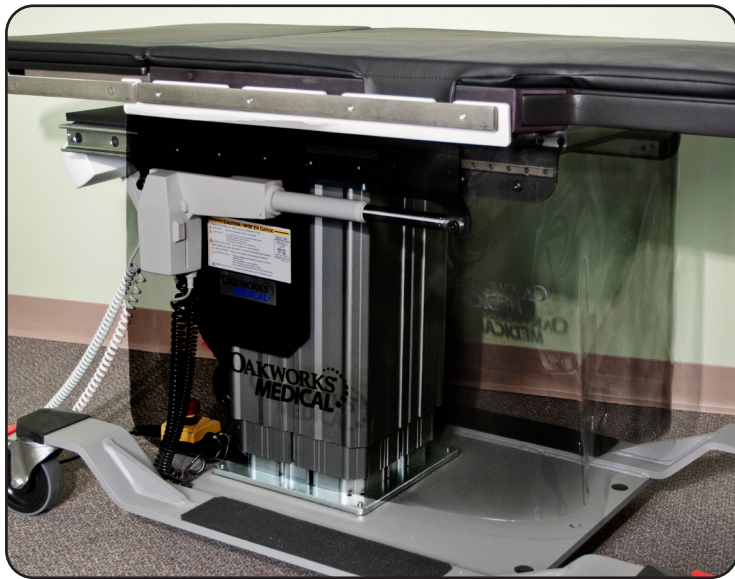


Fig. 4



Fig. 5

CLEANING INSTRUCTIONS

1. Slide one side of the drape attachment off the table's T-Rail support (Fig. 4). Slide the opposite side off and lay the drape on a flat surface with the Oakworks Logo facing up (Fig. 5).
2. Reference the Recommended Cleaners and Disinfectant list (MMINML0008-EN) that came with the product or on our website www.oakworksmed.com under product information.

USER MANUAL: IMAGING AND PAIN MANAGEMENT ACCESSORY - LITHO/UROLOGY TABLE DRAPE CONT...

SYMBOL IDENTIFICATION



This symbol, when used in this manual and on product labels, represents a caution warning. Be sure to read and comply with all precautions and warnings.



This symbol, when used in this manual and on product labels, indicates the name and address of the manufacturer.



This symbol, when used in this manual or on product labels, indicates the country of manufacture along with date of manufacture of the device next to it. Sitting is prohibited in this area.

UNIQUE DEVICE IDENTIFICATION (UDI) INFORMATION

The UDI Label is located on the inside of the left side drape attachment.





Manufacturer
Info

Model Name

Unique Device Information

- GTIN - 14 digit number unique for the product
- Manufacture Date - Country of manufacture and date the device was manufactured in YYYY-MM-DD format
- CE - CE Mark
- SN - Serial Number
- MD - Medical Device Symbol
- QTY - Quantity of the Product

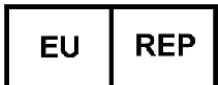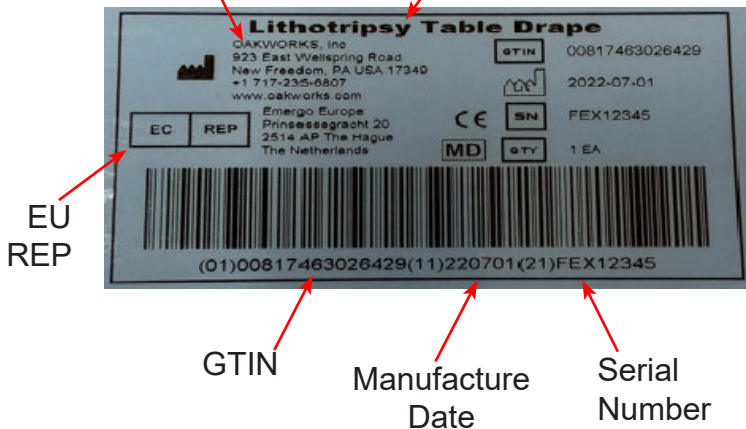

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

CoYoMe B.V.
Edisonstraat 4
3261 LD Oud-Beijerland, The
Netherlands
www.coyome.eu
Enrico Cohen Enrico@coyome.nl
info@coyome.nl
Phone: +31 613.886.424



923 East Wellspring Road
New Freedom, PA USA 17349
Phone: 717-235-6807
www.oakworksmed.com



Part No.: MMINMD0001-EN
Revision: 3
Revision Date: 08/26/2026