USER MANUAL

OAKWORKS®

**Imaging and Pain Management
Accessory - Carbon Fiber Armboard**

USER MANUAL - SINGLE CARBON FIBER ARM BOARD

MODELS

Standard Arm Board:

- 3395-01: with 1" pad (2.5cm)
- 3395-05: with 2" pad (5cm)
- 3395-07: with 3" pad (7.5cm)
- 3395-06: with 4" pad (10cm)
- 3395-04: with 2" & 4" pads (5cm & 10cm)

PART IDENTIFICATION

Standard Arm Board Pads:

- 4405-06: 1" pad (2.5cm)
- 4407-06: 2" pad (5cm)
- 4409-06: 3" pad (7.5cm)
- 4408-06: 4" pad (10cm)

USE

This radiolucent device can be used with any flat support surface where ionizing radiation (x-ray) is utilized for imaging. This arm board can be used for imaging in any environment including private practice, hospital & surgery. It can be used to support arms and feet.

Note: This product is an imaging platform. It is not a surgical platform

DIRECTIONS

Unfold the Arm Board and slide the **Base Section** under the patient and the table top. Position the patient and the **Support Section** to suit the needs of the procedure. The weight of the patient will hold the Arm Board in place.

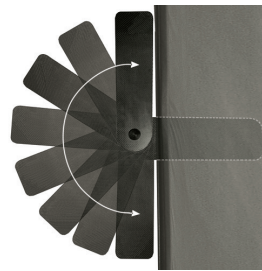
Rotate the Support Section to the desired position. This section rotates up to 180°.

Prone Position at 135°

Supine Position at 180°

⚠ CAUTION

DO NOT exceed 30 lbs. (13.6 kg) of load on the **Support Section**. Injury can occur.

To remove the Arm Board, ask the patient to move their limb off the **Support Section** and lift their body slightly. This will allow you to pull out the **Base Section** of the Arm Board. Fold the **Base and Support Sections** for storage.


CLEANING & DISINFECTION

Reference the Recommended Cleaners and Disinfectant list (MMINML0008-EN) that came with the product or on our website www.oakworksmmed.com under product information.

SPECIFICATIONS

Dimensions (Width x Length) Standard Arm Board	Base Section: 7.5" x 24" (19cm x 61cm) Support Section: 6" x 23" (15cm x 58cm)
Maximum Weight Rating	30 lbs. (13.6 kg)
Radiolucency	Arm Board: Maximum of 1.30mm AL @ 100 kVp. HVL of 3.6mm 1" Pad (2.5cm): Maximum of .42mm AL @ 100 kVp. HVL of 3.6mm 2" Pad (5cm): Maximum of .57mm AL @ 100 kVp. HVL of 3.6mm 3" Pad (5cm): Maximum of .68mm AL @ 100 kVp. HVL of 3.6mm 4" Pad (10cm): Maximum of .79mm AL @ 100 kVp. HVL of 3.6mm

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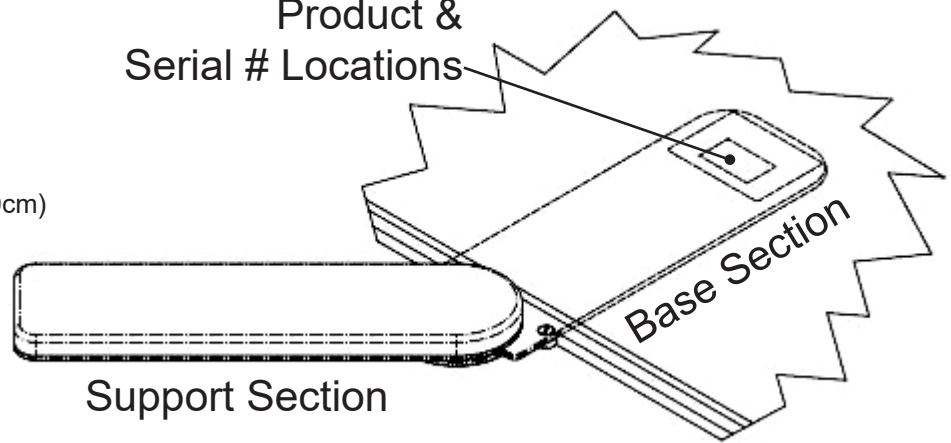UNIQUE DEVICE IDENTIFICATION (UDI) INFORMATION

SYMBOL IDENTIFICATION

This symbol, when used in this manual and on product labels, represents a caution warning. Be sure to read and comply with all precautions and warnings.

This symbol, when used in this manual and on product labels, indicates the name and address of the manufacturer.

This symbol, when used in this manual or on product labels, indicates the country of manufacture along with date of manufacture of the device next to it.

UNIQUE DEVICE IDENTIFICATION (UDI) INFORMATION

The UDI Label is located on the underside of the arm board

Unique Device Information:

- GTIN - 14 digit number unique for each variation of a model
- Manufacture Date - Country of manufacture and date the device goes into production in YYYY-MM-DD format
- CE - CE Mark
- SN - Serial Number
- MD - Medical Device Symbol
- QTY - Quantity of the product

THIS PAGE IS INTENTIONALLY LEFT BLANK

THIS PAGE IS INTENTIONALLY LEFT BLANK

USER MANUAL

OAKWORKS®

Imaging and Pain Management Accessory - Carbon Fiber Armboard


CONTACT INFORMATION:

**OAKWORKS® Inc.**

923 East Wellspring Road
New Freedom, PA USA 17349
Phone: 717.235.6807
FAX: 717.235.6798
www.oakworksmed.com

CoYoMe B.V.

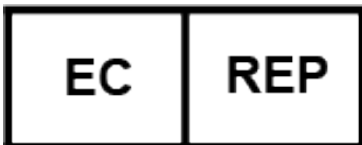Edisonstraat 4
3261 LD Oud-Beijerland, The Netherlands
www.coyome.eu
Enrico Cohen
Enrico@coyome.nl
info@coyome.nl
Phone: +31 613.886.424

EMERGO EUROPE
Westervoortlaan 60
6827 AT Arnhem
The Netherlands

Original Instructions
Manual Part Number MMINMD0003-EN / Med-Eng-PM-84
Revision Level: 7
Revision Date: 06/27/2023

Printed in USA