


USER MANUAL: PRONE PILLOW

PRODUCT USE

The Prone Pillow is designed to comfortably support your clients during a variety of procedures and offers excellent air flow for long prone procedures. The Prone Pillow can be used on any horizontal surface.



PRODUCT DESCRIPTION

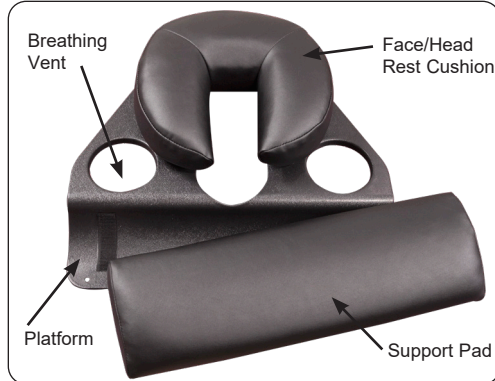

The unit is composed of an easily cleaned, durable polymer platform and a proprietary pad system made up of two removable, positioning pads: the Face/Head Rest Cushion and the Support Pad. These pads are positioned on the platform through the use of Velcro® attachment strips that hold the pads securely in the perfect position for both comfort and procedural need, regardless of client size.

SAFETY INSTRUCTIONS



WARNING

- Not for use with intubated patients.
- For imaging applications - generator must be below the patient and the receiver above the patient.

SPECIFICATIONS

Dimensions)	Platform - 19" x 19" Support Pad - 3" H x 6" W x 19" L Face/Head Rest Cushion - 11" Diameter
Radiolucency	Platform - .44mm AL @ 100 kVp. HVL of 3.6mm Support Pad - .86mm AL @ 100 kVp. HVL of 3.6mm Face/Head Rest Cushion - .72mm @ 100 kVp & HVL of 3.6mm


CLEANING & DISINFECTION

Reference the Recommended Cleaners and Disinfectant list (MMINML0008-EN) that came with the product or on our website www.oakworksmed.com under product information.




USER MANUAL: PRONE PILLOW

INSTRUCTIONS FOR USE - PRONE POSITIONING

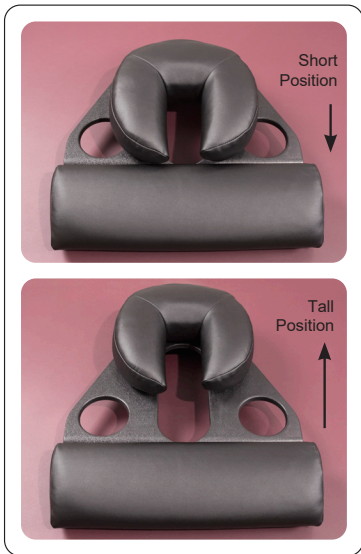


The Face/Head Rest Cushion can be positioned anywhere along the length of the large center vent to accommodate clients of different heights. The width of the cushion's open end can be adjusted before placing on the platform to fit the different sized faces. Adjust the Face/Head Rest Cushion placement for comfortable support by positioning it along the Velcro strips so that the head/face and shoulders are equally supported.

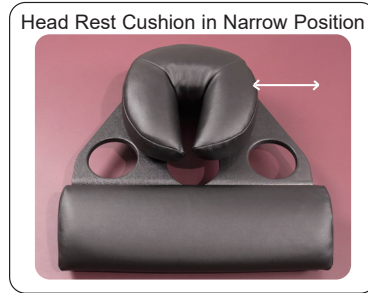

INSTRUCTIONS FOR USE - SUPINE POSITIONING




The Face/Head Rest Cushion can be used to facilitate comfort and support when the client is lying in a supine position. Turn the cushion around on the platform so that the open end points away from the client. This angle provides cervical support to the client when in a supine position.

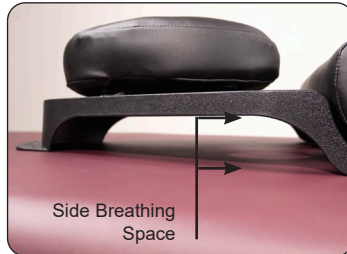

NOTE: The Prone Pillow should always be set high enough so that there is sufficient air flow. When using the Prone Pillow on soft surfaces for prone positioning, it is important to insure the air flow not be restricted in any way and that the client can breathe freely. When using the platform on a soft horizontal surface - such as a bed - it is recommended that a firm, flat surface/object be placed under the platform to lift it above the padding and allow good air flow to the client.

ACCESSORIES

Disposable Face/Head Rest Covers: A disposable sanitary covering perfectly sized to do a superb job of making your clients more comfortable while also protecting the Prone Pillow. Choose from fitted or flat.

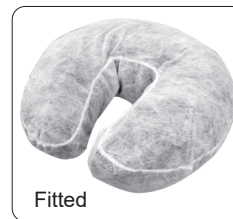

Flannel Face/Head Rest Covers: Machine-washable, unbleached 100% cotton flannel cover fits securely with elastic and provides added comfortable protection.



USER MANUAL: PRONE PILLOW

SYMBOL IDENTIFICATION



This symbol, when used in this manual and on product labels, represents a caution warning. Be sure to read and comply with all precautions and warnings.



This symbol, when used in this manual and on product labels, indicates the name and address of the manufacturer.




This symbol, when used in this manual or on product labels, indicates the country of manufacture along with date of manufacture of the device next to it.

UNIQUE DEVICE IDENTIFICATION (UDI) INFORMATION



The UDI Label is located on the underside of the platform

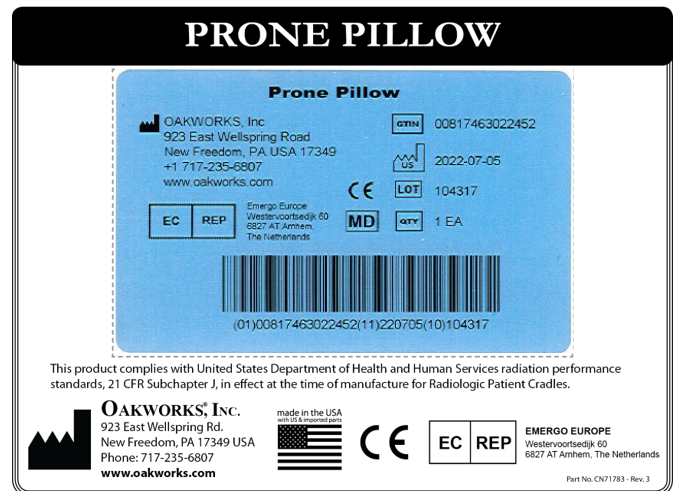

Manufacturer Information

Model Name



Unique Device Information

- GTIN - 14 digit number unique for each variation of a model
- Manufacture Date - Country of manufacture and date the device goes into production in YYYY-MM-DD format
- CE - CE Mark
- LOT - Lot Number
- MD - Medical Device Symbol
- QTY - Quantity of the Product

EU Representative

GTIN

Manufacture

Serial Number

Date



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

CoYoMe B.V.
Edisonstraat 4
3261 LD Oud-Beijerland, The Netherlands
www.coyome.eu
Enrico Cohen Enrico@coyome.nl
info@coyome.nl
Phone: +31 613.886.424



923 East Wellspring Road
New Freedom, PA USA 17349
Phone: 717-235-6807
www.oakworksmed.com



Part No.: MMINML0012-EN
Revision: 3
Revision Date: 05/21/2025